



1. Product Name

Bouncy Lily Pads – Mini Lily Pads – Set of 4

2. Product Code

21569

3. Colour

Colours may vary

4. Brief Description

With heavy duty springs distributed evenly between two layers of Baltic birch wood.

5. Contents

- 4 x Mini Lily Pads

6. Snoezelen® Stimulations

- Movement

7. Best Use

Use on a flat, stable floor. Use with the Large Lily Pad to encourage movement onto targets at a variety of heights and diameters. These Lily Pads can be used as part of sensory circuits to prepare learners for classes at the start of the school day. Remove shoes before use.

8. Compatible ROMPA® Products

21563 – Large Lily Pad

21571 – Bouncy Lily Pond Set

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

With heavy duty springs distributed evenly between two layers of Baltic birch wood. Completely covered in easy to clean polyurethane vinyl to prevent hands and feet getting pinched. With a non-slip rubber base for added safety. A great additional to an obstacle course or sensory circuit. Use to encourage use of the imagination as well as using core strength and balance skills. With a stiff bounce, this smaller size is ideal for working on postural control, coordination and mobility training. The heavy duty springs allow consistent pressure and recoil. Colours may vary.

11. Safety

- Always supervise the use of this product
- Do not exceed the working load of 170 lb.
- For indoor use only.
- Use on a flat, stable surface.
- We recommend the use of appropriate mats with these Lily Pads.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Working load: 77 kg (170 lbs)

Construction: Baltic birch wood, polyurethane vinyl, metal springs, non-slip rubber base

13. Installation

1. Allow plenty of unobstructed space around your Lily Pads. Make sure your area is free of hazards/obstacles.
2. Use on a flat, stable floor.
3. Choose a safe yet appropriately challenging distance between the Lily Pads.

14. Care and Maintenance

1. All the parts of your Lily Pads should be checked on a regular basis (at least monthly, and more if the Lily Pads are used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear and replace these as appropriate.
3. Use and store this product indoors, in a safe place.
4. Wipe the Lily Pads clean with a soft, slightly damp cloth. Allow to air dry thoroughly before use. Do not immerse in water.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

