



1. Product Name

Safety Snap

2. Product Code

21570

3. Colour

Likely to be as shown

4. Brief Description

Provides a connection between equipment and either an accessory and/or the final point of suspension.

5. Contents

1 x Safety Snap

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use as part of a sensory integration suspension system with compatible products.

8. Compatible ROMPA® Products

ROMPA® has a range of Sensory Integration equipment that will facilitate movement, proprioception and vestibular input. Please see www.rompa.com for our full range. Compatible products include:

21580 - Theragym® Sling Swing

21581, 21583 - Over the Moon Swing Sets

21579 - Ultimate T Swing

21566 - Adjustable Angle Swing with foot rests

9. Starting Up

All hanging equipment requires a minimum ceiling height of 8 feet/2.5metres and should be positioned a clear distance of 6 feet/1.8metres from adjacent walls and obstacles. All hanging equipment should be used with mats and cushioning of an appropriate thickness as well as helmets.

10. Detailed Description

Provides a connection between equipment and either an accessory and/or the final point of suspension. Screws closed. Steel and zinc-plated. Working load: 225kg (500 lbs).

11. Safety

- Always supervise the use of this product
- Do not use suspension equipment outside.

12. Technical Specification

Material: Steel, then zinc plated

Working Load: 225kg (500 lbs)

Size: 11cm long, approximately 6cm wide; 1cm thick steel

14. Care and Maintenance

- All parts of the safety snap should be checked on a monthly basis and a record kept on the safety sheet provided.
- Worn parts should be replaced immediately.
- On the safety snap, check for wearing on the top and bottom of the snap where it rubs against the fixed point of motion or attached suspended equipment.
- *Note: if wear exceeds 30% of the diameter of the safety snap do not use – replace immediately.*
- Ensure that the self-closing gate closes smoothly and all pins are secure.
- Check the fixed point of suspension (not included) for wear regularly and refer to the instructions for this for guidance on when to discontinue use and replace.
- Store out of reach in a safe place when not in use.
- Wipe with a dry cloth

15. Troubleshooting

- Discontinue use if any mechanisms are stiff, or the item is worn and replace.

Further copies of these instructions can be downloaded at www.rompa.com We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

