



*Shown with 21585 Therapy Net Deluxe Swing*

## **1. Product Name**

Deluxe Net Swing Accessory Board

## **2. Product Code**

21586

## **3. Colour**

Likely to be as shown

## **4. Brief Description**

Use with the Therapy Net Deluxe Swing.

## **5. Contents**

- 1 x board

## **6. Snoezelen® Stimulations**

- Vestibular

## **7. Best Use**

Use with the Therapy Net Deluxe Swing and a safe and appropriate suspension system.

## **8. Compatible ROMPA® Products**

21585 – Therapy Net Deluxe Swing

21928 – Hanging Bracket

21548 – Height Adjustment System

21559 – Safety Rotational Device

21570 – Safety Snap

## **9. Starting Up**

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

## **10. Detailed Description**

Use with the Therapy Net Deluxe Swing. Vinyl-covered birch wood. The complete swing provides linear and rotational movements to enhance core development and motor control. A heavy-duty nylon cord unit is suspended from welded steel rings.

## **11. Safety**

- Always supervise the use of this product
- Always use a Rotational Device to help prevent wear of the swing and the suspension points.
- Do not exceed the working load of 225kg (500 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## 12. Technical Specification

Size: 58cm diameter  
Working load: 225 kg (500 lbs)

## 13. Installation

1. Place the Deluxe Net Swing Accessory Board carefully and centrally inside the net.



2. Please refer to the separate instructions for the Therapy Net Deluxe Swing 21585.

## 14. Care and Maintenance

1. All the parts of this swing should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check the board for signs of wear and tear and replace as appropriate.
3. Check your suspension system for signs of wear and tear and replace as appropriate.
4. Use and store this product indoors, in a safe place.
5. Wipe clean with a slightly damp cloth and a mild cleaning solution.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)



