

**1. Product Name**

Scooter Board Attachment

**2. Product Code**

21602

**3. Colour**


Natural wood finish

**4. Brief Description**

Simply hooks onto the ladder bars of the Climb and Go – Set of 2 (21554) for a scooter board ramp or a wooden slide.

## 5. Contents

- 1 x Ramp Slide
- 1 x Scooter Board Attachment
- 1 x pack of fixings

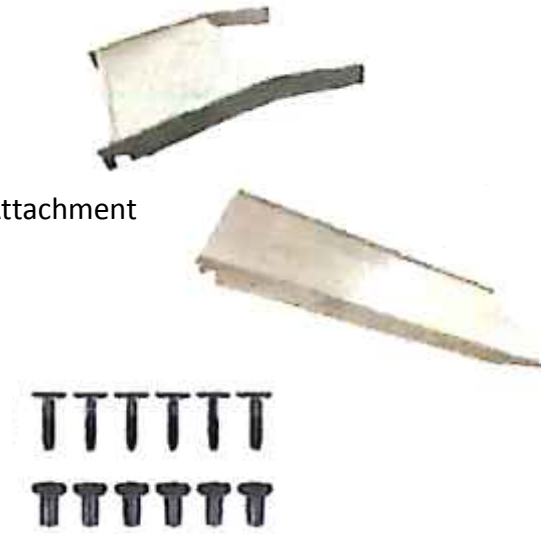

*The product is shown above with one of the units in the Climb and Go Set and a scooter (neither included). The product is also shown below with a wall-mounted gym ladder – again, this is not included – it is for illustration purposes only.*

## 6. Snoezelen® Stimulations

- Balance

## 7. Best Use

For use by a suitably qualified professional, in a safely-located and appropriate place.  
For indoor use only.

## 8. Compatible ROMPA® Products

Please visit [www.rompa.com](http://www.rompa.com) for details about our full range, including wall and floor cushions.

- 21554 Climb and Go – Set of 2
- 12658 ROMPA® Mat
- 14589 Basic Roller Board
- 20050 Round Roller
- 18363 Square Scooter
- 17812 Connect-a-Scooter
- 13734 Adjustable Scooter Board with Tow Strap by ROMPA®
- 16018 Puzzle Block (to knock down)
- 18354 Beginners Bowling Set (to knock down)

## 9. Starting Up

There are no short-cuts to setting up Sensory Integration products. Read all instructions thoroughly before installing and using the Scooter Ramp Attachment and keep the instructions in a safe place.

## 10. Detailed Description

Simply hooks onto the ladder bars of the Climb and Go – Set of 2 (21554) for a scooter board ramp or a wooden slide.

Working load: 113kg (250lbs)

## 11. Safety

### **Always supervise the use of this product.**

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

- Do not exceed the 113kg (250lbs) working load.
- This equipment is intended for professional clinical use only under supervised conditions by a trained specialist. Use without proper supervision or knowledge is potentially dangerous and should not be undertaken or permitted. Before using these products, make sure you know your limitations and that of your equipment.
- Client safety is dependent on the appropriate choice of activity to match, yet challenge physical ability.
- The layout of the activity area, proper installation and maintenance of equipment is critical – use this product appropriately and safely.
- Do not use or store outside.
- Take time to familiarize yourself with the use and maintenance of this equipment before using and file these instructions in an accessible location. These instructions must be read and be familiar to anyone who is using this equipment.
- All parts of the product should be thoroughly checked at least on a monthly basis and before use.
- Check all points where movement occurs for wear of any kind. All wear points should be checked regularly and recorded on the maintenance checklist enclosed to monitor their condition. Worn parts should be replaced or repaired immediately.
- Clients should wear a safety helmet as necessary.
- Floor mats of an appropriate thickness should be positioned properly underneath and around all equipment. Wall matting/cushioning should also be considered if appropriate.
- Risk assess before use. Inspect equipment for wear and tear before use and discontinue use if any doubt.

## 12. Technical Specification

Size: 202L x 66.5W x 9cmH

Working load: 113kg (250lbs)

## 13. Installation

- Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.
1. Attach the top platform piece to the slide ramp piece.



2. You will need a 4mm (5/32") allen key (not included) to tighten your screw sets. Place the open end of the screw pair on the inside of the ramp, and screw in the threaded piece from the outside. Repeat for all six holes.
3. Double-check that all hardware is securely tightened before use (first use and every use).
4. Make sure that the unit you are attaching the Scooter Board Attachment to (e.g. 21554 Climb and Go – Set of 2 or a wall-mounted frame intended for balance beams, PE equipment etc of a comparable weight) is on a flat and level surface. Attach the Scooter Board Attachment to the desired rung by hooking both the top platform end hooks securely over the rung.

## **14. Care and Maintenance**

1. Wipe down with a soft cloth dampened with a very mild soap solution and allow to air-dry naturally. Do not use large amounts of liquid and do not immerse any part of this product in water. If this product were exposed to wet conditions or left outside, the moisture would cause the wood to swell and break apart.
2. Use and store in a dry environment.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

## SENSORY INTEGRATION MAINTENANCE CHECK LIST

**PRODUCT:** .....

[illegible]

