



1. Product Name

Roller Slide Attachment

2. Product Code

21605

3. Colour


Likely to be as shown

4. Brief Description

You're in for a bumpy ride! Simply hooks onto the ladder bars of the Climb and Go – Set of 2 (21554) as a horizontal bridge (using both) or as a slide (using just one).

5. Contents

- 1 x Roller Slide Attachment (already assembled, with 6 bars)
Shown above with one of the units in the Climb and Go Set (not included)

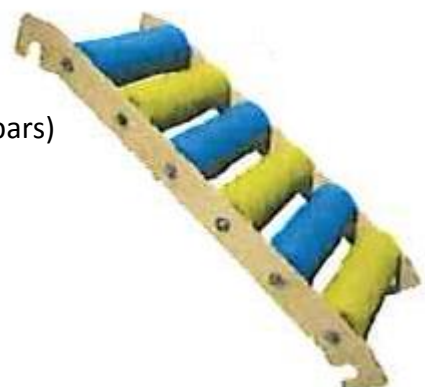

6. Snoezelen® Stimulations

- Touch

7. Best Use

For use by a suitably qualified professional, in a safely-located and appropriate place.
For indoor use only.

8. Compatible ROMPA® Products

Please visit www.rompa.com for details about our full range.

- 21554 Climb and Go – Set of 2
- 12658 ROMPA® Mat

9. Starting Up

There are no short-cuts to setting up Sensory Integration products. Read all instructions thoroughly before installing and using the Roller Slide Attachment and keep the instructions in a safe place.

10. Detailed Description

You're in for a bumpy ride! Simply hooks onto the ladder bars of the Climb and Go – Set of 2 (21554) as a horizontal bridge (using both) or as a slide (using just one). Offers sensory stimulation (vibration), strengthening practice, bilateral coordination and core stability. Rollers are vinyl-covered.
Working load: 90kg (200 lbs)

11. **Safety**

Always supervise the use of this product.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

- Do not exceed the 90kg (200lbs) working load.
- This equipment is intended for professional clinical use only under supervised conditions by a trained specialist. Use without proper supervision or knowledge is potentially dangerous and should not be undertaken or permitted. Before using these products, make sure you know your limitations and that of your equipment.
- Client safety is dependent on the appropriate choice of activity to match, yet challenge physical ability.
- The layout of the activity area, proper installation and maintenance of equipment is critical – use this product appropriately and safely.
- Do not use or store outside.
- Take time to familiarize yourself with the use and maintenance of this equipment before using and file these instructions in an accessible location. These instructions must be read and be familiar to anyone who is using this equipment.
- All parts of the product should be thoroughly checked at least on a monthly basis and before use.
- Check all points where movement occurs for wear of any kind. All wear points should be checked regularly and recorded on the maintenance checklist enclosed to monitor their condition. Worn parts should be replaced or repaired immediately.
- Clients should wear a safety helmet as necessary.
- Floor mats of an appropriate thickness should be positioned properly underneath and around all equipment. Wall matting/cushioning should also be considered if appropriate.
- Risk assess before use. Inspect equipment for wear and tear before use and discontinue use if any doubt.

12. **Technical Specification**

Size: 161L x 66W x 14.5cmH

Working load: 90kg (200lbs)

Materials: Phthalate-free

13. Installation

- Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.
1. Make sure that the unit you are attaching the Roller Slide Attachment to (e.g. 21554 Climb and Go – Set of 2 or a wall-mounted frame intended for balance beams, PE equipment etc of a comparable weight) is on a flat and level surface. Attach the Roller Slide Attachment to the desired rung by hooking both the Roller Slide end hooks securely over the rung.



14. Care and Maintenance

1. Wipe down with a soft cloth dampened with a very mild soap solution and allow to air-dry naturally. Do not use large amounts of liquid and do not immerse any part of this product in water. If this product is exposed to wet conditions or left outside, the moisture will cause the wood to swell and break apart.
2. Use and store in a dry environment.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

SENSORY INTEGRATION MAINTENANCE CHECK LIST

PRODUCT:

[illegible]