



1. Product Name

TheraGym® Soft and Plush SensiMats – Set of 2

2. Product Code

21606

3. Colour

The shade may vary, but likely to be brown

4. Brief Description

Enhance your TheraGym® Square Platform Swing with these special mats that provide multiple tactile and visual sensory experiences.

5. Contents

- 1 x suede mat
- 1 x long-pile carpet mat

6. Snoezelen® Stimulations

- Touch

7. Best Use

For use with the Square TheraGym® Platform Swing

8. Compatible ROMPA® Products

21549 Square TheraGym® Platform Swing

See www.rompa.com for helmets, mats and a large range of Sensory Integration products

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Enhance your TheraGym® Square Platform Swing with these special mats that provide multiple tactile and visual sensory experiences.

11. Safety

- Always supervise the use of this product
- The carpet has long fibres which are not suitable for those who mouth.

Swings and their accessories are not toys. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Size: (each) 66 x 66cm



13. Installation

Please also refer to the separate instructions for the Platform Swing.



Use one carpet at a time.

Line up the carpet on the swing.

There are four hook and loop (Velcro®) fasteners attached to the corners of the carpet tile.

Detach the plastic adhesive covering from all four hook and loop (Velcro®) fasteners, and position the carpet piece gently on the Platform Swing. Once the carpet tile is centred, press firmly on the corners to affix the hook and loop (Velcro®) squares and to set the adhesive to the wood.

To interchange the carpet pieces, GENTLY pull the hook and loop (Velcro®) pieces apart. Replace with the other carpet tile.

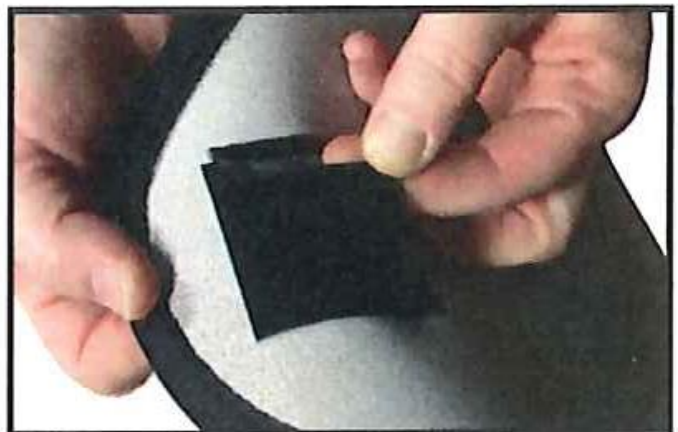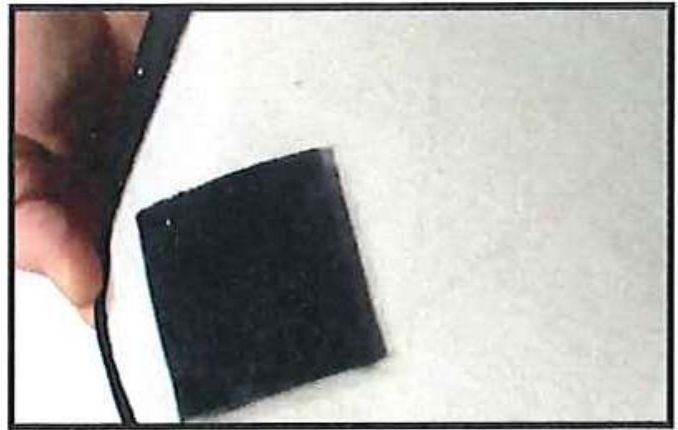

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate.
3. Use and store this product indoors, in a safe place. Do not allow this product to become wet.
4. Remove the tile from the swing and shake to remove any dust. Spot clean any marks with a damp cloth and a mild cleaning solution. Do not immerse the tiles in water.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com

