1. Product Name

Shimmering Curtain

2. Product Code

21748

3. Colour


White frame

4. Brief Description

A spectacular shimmering curtain of fibre optics which continually change colour

5. Contents

- 1 x curtain assembly with grub screws
- 2 x mounting brackets
- 1 x Allen key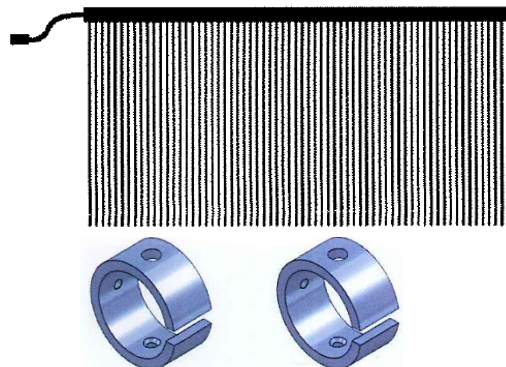

This product requires, but does not include, a light source 19210.

6. Snoezelen® Stimulations

- Sight

7. Best Use

Use with a light source 19210. The Side Glow Fibre Optic Spray changes colour constantly along its length to the tip, where the light is brightest. The changing colours are visually stimulating and encourage focusing, yet are calming and relaxing too. The strands also provide fascinating tactile stimulation. Some suggestions for use are:

1. Stroke the strands to demonstrate that it is safe to touch them.
2. Fan out the bright tips to cast a coloured light across a hand or face.
3. Carefully wrap the spray round limbs or the whole body (not around the neck).
4. Use interactively with our Wi Fi switching system.

8. Compatible ROMPA® Products

19210 – Light Source

9. Starting Up

- Secure the Light Source e.g. on wall or shelf out of reach of clients.
- Attach the curtain to a wall or doorway using the fixings included in an appropriate place.
- Slide the boss into the metal tubing on the front face of the light source and secure in place. Plug in the light source at the wall. Switch the light source on

10. Detailed Description

A spectacular shimmering curtain of fibre optics which continually change colour. Place at the entrance of your SNOEZELLEN® or attach to the wall for visual stimulation. The 66 plastic fibre optic strands should not be vigorously pulled. Requires light source 19210.

11. Safety

- Always supervise the use of this product

NOTE: Fibre optic products are a constant source of safe tactile and visual stimulation but they should always be used under professional supervision bearing in mind the following important points:

- Install the light source out of reach of clients.
- Though safe to stroke across the face, the strands should never be put in the mouth.
- Should you wish to tie back a curtain (e.g. to clean the floor underneath it or to move something large through it without obstruction) use a temporary tie back that will undo or break easily (e.g. a paper chain or a very loosely tied short cord. This way, should someone get himself/herself caught or ravelled in the fibre optics they can easily and quickly become free.


12. Technical Specification

Size: 2mL x 1mW
Strands: 66
Weight: 2kg


13. Installation

- To avoid damaging the fibre optics, you may wish to install the Curtain so that the ends of the fibre optics cannot be trodden on (i.e. they are not reaching the floor).
- Please refer to the separate instructions for the Light Source. Place the Light Source on wall or shelf. Make sure that the Light Source will remain in place and will not fall. Place the Light Source out of reach of clients. Permanently fix the light source in place if necessary.
- Attach the curtain to a wall or doorway. There are two rings included. Please purchase appropriate screws, fixings etc. These depend entirely on the surface the curtain frame is attached to.
- Mark the positions on your surface for the two mounting brackets (ring-shaped fixings). These fixings should be around 50mm from each end of the wall tube. Drill holes to take wall plugs for suitable screws/bolts. Having chosen the appropriate fixings and you are sure that the surface will comfortably hold the weight of the product, fix the brackets in place, making sure the cut out (gap in the ring) is facing the floor.


**Tighten screw fully
through hole in back
and into wall**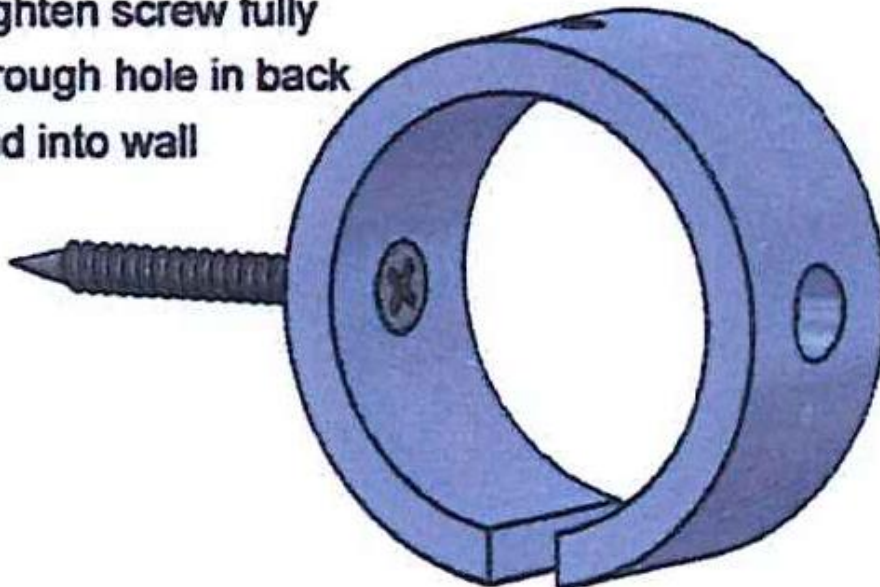

- Slide the tube through the mounting brackets. The fibre optic strands will slide through the cut out in the bottom of the ring.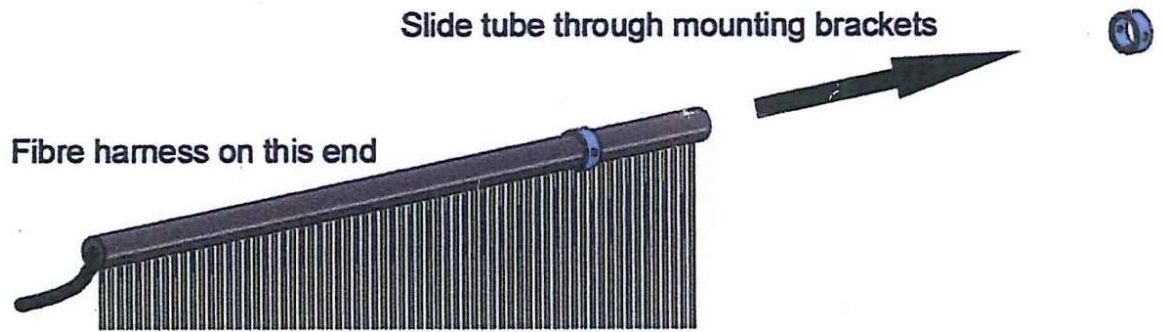

- Secure the tube to the brackets by tightening the supplied grub screws into the top holes on the brackets using the supplied Allen Key.

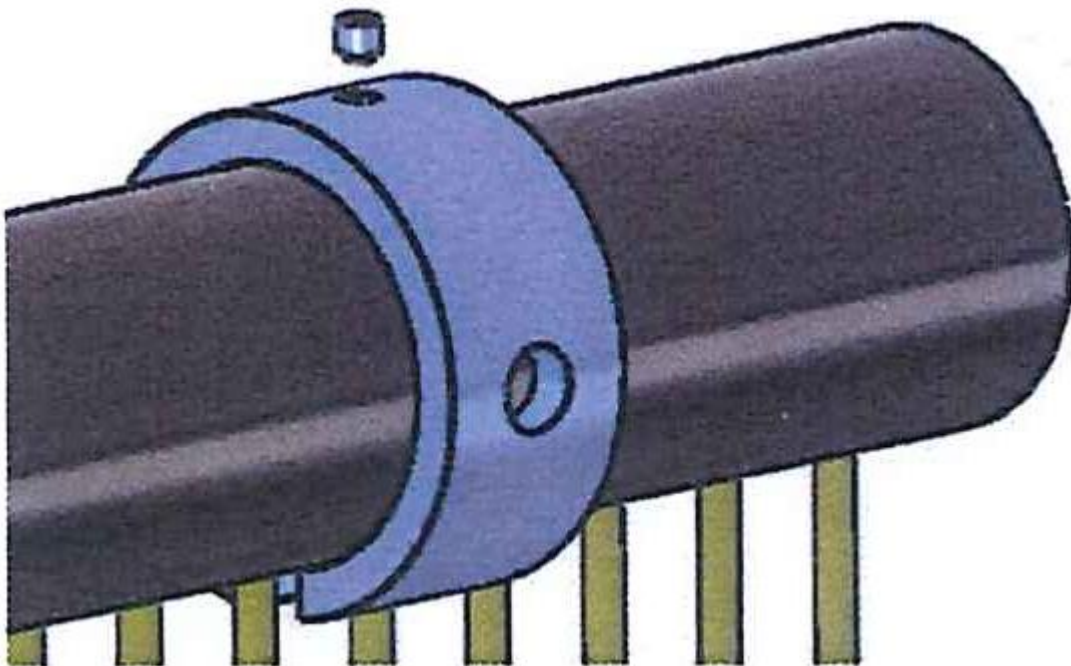- Slide the boss into the metal tubing on the front face of the light source and secure in place. Plug in the light source at the wall. When using this with fibre optics, it may be necessary to adjust the position of the boss in the port (i.e. not completely pushed in as far as possible) to allow as many strands as possible to be illuminated. Switch the light source on.

14. Care and Maintenance

Fibre Optic products provide dramatic visual and tactile stimulation. However, these products should always be handled under close adult supervision. If misused or used carelessly they could cause injury. ROMPA® cannot accept any liability for injury or damage caused by the abuse or misuse of these products.

- Your Fibre Optics should always be used in areas away from sharp edges.
- The use of Fibre Optic products is not recommended where there is a risk of accidental ingestion. Fibre Optics must not be placed in the mouth.
- Do not pull on the curtain. Do not place excessive strain on the fibre optic strands or on the light source cable.
- Clean the fibre optic strands using a mild detergent and a slightly damp cloth, taking care not to wet the boss. Dry thoroughly before re-use. Do not immerse the fibre optics in water.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

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