



**1. Product Name**

Trapeze Swing

**2. Product Code**

21773

**3. Colour**


As shown.

**4. Brief Description**

Includes the trapeze bar with two straps and two Safety Snaps.

Sensory Integration is a therapy that should be carried out by suitably trained and qualified people. These swings are intended for controlled movements - they are not toys.

## 5. Contents

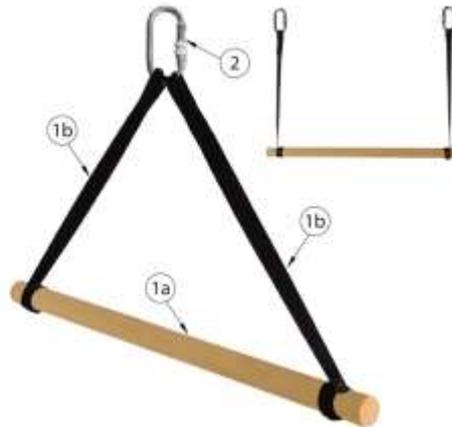

- 1 x trapeze bar (1a)
- 2 x straps (1b)
- 2 x safety snaps (2)

## 6. Snoezelen® Stimulations

- Vestibular

## 7. Best Use

Use from a safe and appropriate suspension system.

## 8. Compatible ROMPA® Products

21928 – Hanging Bracket  
21559 – Safety Rotational Device  
21570 – Safety Snap

ROMPA® offers an extensive range of helmets, mats and a large range of SI equipment and sensory resources. Visit [www.rompa.com](http://www.rompa.com) for more information.

## 9. Starting Up

- Unpack and attach safety as per instructions to an appropriate and safe suspension system.

## 10. Detailed Description

## 11. Safety



Please read all the following safety information before each use

### **Always supervise the use of this product**

Always use the Rotational Device to help prevent wear of your swing and the suspension points.

- Do not exceed the working load: 225kg (500 lbs).
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## 12. Technical Specification

|                |                |
|----------------|----------------|
| Size (bar):    | 76cmL          |
| Size (straps): | 91cm           |
| Working Load:  | 225kg (500lbs) |

## 13. Installation

Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.

**Product Installation / Assembly:**

- Turn locking nut clockwise to its fullest extent to allow the **Safety Snap** to open.



- Turn locking nut counterclockwise to its fullest extent to securely lock.



- Each **Safety Snap** is intended for use with one (1) piece of equipment and / or one (1) accessory maximum.



- Be sure all hanging equipment is clear of other obstacles (a 6' clearance around any hanging item is recommended), and test the unit for security and safety before use (minimum ceiling height of 8' is recommended).

- With the threaded loop to the inside of the hole in the bar, pinch the open loop end of the strap (2) and thread it through the hole in the bar (3).



- Pull the strap all the way through the hole in the bar and then slide the threaded loop to the outside of the bar so that it touches, but can't go past, the hanging strap (4). Pull the hanging strap straight and attach to the half-moon base with a **Safety Snap**. Repeat for the other strap on the other side of the bar.

The Trapeze bar already comes assembled. The following is how to replace the reinforced nylon straps.

- To attach trapeze bar:



- Open the loop on one end of the strap and slide it on to one end of the trapeze bar (1), sliding it towards the middle of the bar just past the hole at the bar's end.



## 14. Care and Maintenance

- All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
- Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the ropes and straps for loose stitching.
- Use and store this product indoors, in a safe place.
- Dust or wipe the components clean with a damp cloth. Do not immerse in water. Allow to dry before use

## 15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)  
We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)

