



### **1. Product Name**

LED Mood Lamp

### **2. Product Code**

21888

### **3. Colour**

White exterior

### **4. Brief Description**

Lights up your multi-sensory environment to suit your preferences. Choose soft pastel colours or powerful and intensive colours.

### **5. Contents**

- 1 x lamp
- 1 x USB cable

### **6. Snoezelen® Stimulations**

- Sight

## 7. Best Use

Use in a darkened environment.

## 8. Compatible ROMPA® Products

*Not applicable*

## 9. Starting Up

- Connect the USB cable into the socket at the rear of the lamp.
- Connect the other end of the cable to a USB port and charge for around 1.5 hours.

## 10. Detailed Description

Lights up your multi-sensory environment to suit your preferences. Choose soft pastel colours or powerful and intensive colours. For example, choose pastel green for relaxation ... powerful yellow to stimulate and make you feel happy ... or bright white for rest. An appropriate light for every mood as well as individual taste and tolerance. The acrylic diffuser screen spreads the beautiful light around the room. Stylish design. White exterior encourages focus on the effect created rather than on the unit itself. Battery-operated – charge using the provided USB cable for approx. 1.5 hours. Operates for around 2.5 hours once charged.

## 11. Safety

- Always supervise the use of this product.
- Dispose of any packaging appropriately.
- This product is for visual purposes only and is not intended to be handled.
- This product is not a toy.
- Make sure the light is not pointing into people's eyes – do not look directly into the light.

## 12. Technical Specification

Size:	9cmL x 8cm diameter
Weight:	0.156 kg
Input voltage:	5 V DC 500m
Power:	Li-on Battery 3.7V DC 280mAh
Colours:	The battery is built in and cannot be replaced 256 hues



## 13. Installation

1. Choose a safe, suitable and dry indoor location for your mood lamp.
2. Use and store this lamp away from extreme temperatures, direct sunlight, high humidity, knocks and scrapes, moisture, flammable gases, vapours and solvents.
3. Sit the lamp on its small, black, circular integral stand.
4. Fully charge the lamp using the USB cable provided before initial use – connect the round pin plug of the USB charger to the power socket on the back of the lamp. Connect the USB plug of the USB charger into a suitable USB port on your computer. Once the power supply has been established, the battery charge indicator lights up (see the back of the lamp). The battery charge indicator will go out when the battery is fully charged. A complete recharge takes around 2 hours. Disconnect the USB cable once charging is complete.  
**IMPORTANT:** Never directly touch the touch sensor (coloured bar) after charging – always touch the ON/OFF sensor first (end of the bar).  
**IMPORTANT: Do not touch the touch sensor while pressing the RESET button.**  
The lamp will operate for around 2.5 hours once charged.
5. Press the ON/OFF sensor to turn the product on. Touch the ON/OFF sensor for 3 seconds to activate the continuously colour changing mode.
6. Manually select the desired colour using the bar across the top of the lamp.
7. After use, to switch the product off, touch the ON/OFF sensor.

## 14. Care and Maintenance

- Keep dry and away from moisture – keep and use indoors.
- Do not strike the product – handle it with appropriate care. Dropping the lamp, even from a short distance, is likely to cause irreparable damage.
- This product contains LEDs so there will be no need to change any bulbs or lamps. There are no user-serviceable parts to this lamp.
- Do not place the lamp under any mechanical stress.
- Do not damage the rechargeable battery or its casing.
- Never short-circuit the contacts of the rechargeable battery.
- Never throw the product or its battery into fire.
- Charge the rechargeable battery regularly, even if you are not using the product. It is not necessary to discharge the rechargeable battery before recharging.
- Once this product is eventually exhausted, please dispose of it appropriately. Case use of the lamp if it is visibly damaged, has ceased working or has become exposed to moisture or extreme heat.

## 15. Troubleshooting

In the unlikely event of problems, press the RESET button at the back using a thin and blunt object. Make sure the unit is fully charged.

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)