



1. Product Name

Tube Swing

2. Product Code

21893 – Child

21894 – Maxi

3. Colour

As shown

4. Brief Description

Multi-directional swing that is colourful and appealing. Sit on it or lie across it on your tummy. Fun and functional.

5. Contents

- 1 x inner tube
- 1 x multi-coloured cover
- 1 x locking safety snaps
- 2 x ropes (supplied knotted together)

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use from a safe and appropriate suspension system.

8. Compatible ROMPA® Products

21928 – Hanging Bracket

21559 – Safety Rotational Device

23011 (21570, 21590) – Safety Snap

21567 – Vertical Bouncer Kit

See www.rompa.com for mats and a large range of Sensory Integration products

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

Multi-directional swing that is colourful and appealing. Sit on it or lie across it on your tummy. Fun and functional. The heavy-duty and zipped polyamide cover is both inviting and practical as this covers the air valve of the inner tube and makes the swing easier to hold onto. Includes reinforced suspension bands.

11. Safety

- Always supervise the use of this product
- Always use the Rotational Device to help prevent wear of your swing and the suspension points.
- Do not exceed the maximum weights supported – 80kg for the Child version, and 100kg for the Maxi version.
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Child

Maximum weight supported: 80kg

Size: 50cm internal diameter; 85cm external diameter

Maxi

Maximum weight supported: 100kg

Size: 83cm internal diameter; 135cm external diameter

Cover: 100% polyamid with a zipper closure

Inner tube: rubber and so likely to contain latex

13. Installation

Allow plenty of unobstructed space around your swing. Suspended equipment needs a minimum ceiling height of 2.5m (8 feet). Suspended equipment should be positioned a clear distance of 1.8m (6 feet) from walls and obstacles.

Safety Snap

The Safety Snap included can be locked. Turn the barrel to move it up and down and so open and close the Safety Snap. The Safety Snap shown to the right and below is slightly different, but the principle is the same.



Tube Swing

1. Open the zipper of the cover and open out the cover.
2. Place the black inner tube on the open cover as it may be easier to inflate the tube with the cover underneath rather than trying to fit the cover onto an inflated tube.
3. Locate the inflation valve on the black inner tube. This is a small hole (no separate parts or protruding stem), with a circle around it to help you to find it. Insert the needle of your pump into this hole and inflate the inner tube. Inflate the tube gradually in an environment that is at room temperature (not a cold room). Inflate the tube to around 80% full and then top this up gradually.
4. Close the zip all the way around the cover. The start and end of the zipper will be at the top of the swing.
5. The black straps (the reinforced bands) are already attached to the cover. These have a loop at each end of each strap – place these together and join these using the

locking safety snap. The included ropes could be used to extend the length of the black straps but the use of these may not be necessary (the straps may be long enough as they are or you may prefer to use a Height Adjuster). The ropes have slits woven into them – these can be used to vary the length of the black straps.

6. Connect the locking safety snap to your point of suspension and lock the safety snap closed.

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the cover for loose stitching.
3. Use and store this product indoors, in a safe place.
4. Dust or wipe the components clean with a damp cloth. Do not immerse in water. Allow to dry before use.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com

