**Mega Pod™
21905**

This product is Windows-based and allows you to choose an appropriate Windows Operating System password for your own security if you wish.

If you choose to apply one, please choose a password carefully, making sure that you and any colleagues who will also need this, have a failsafe way of remembering the password.

Please be aware that recovery of forgotten passwords in these products is not straight-forward nor speedy.

Password recovery for Windows is not covered under the terms of our standard customer support, and as such is a chargeable service.

Should you require additional information please contact ROMPA®.

1. Product Name

Mega Pod™

2. Product Code

21905

3. Colour

As shown

4. Brief Description

9 interactive applications in one panel! Easy to use touch screen technology.

5. Contents

- 1 x Panel
- 1 x universal adapter (UK/European/USA)
- 1 x wireless keyboard and trackball unit

6. Snoezelen® Stimulations

- Sight
- Sound

7. Best Use

Use in a darkened environment, for maximum visual impact.

8. Compatible ROMPA® Products

20551 - Microphone

9. Starting Up

- Wall mount
- Plug in and switch on

10. Detailed Description

9 interactive applications in one panel! Easy to use touch screen technology. Exciting applications include: Kaleidoscope, Light Speed, Trace-A-Shape, Music Studio and Fireworks. All applications are fully interactive by touching the screen. Modify each application easily for the user's ability.


High-impact acrylic protective panel over a 37" LCD display, built in speakers, external jack for handheld microphone and external USB ports for accessories and software upgrades.

Applications include:

- Kaleidoscope
- Spiral Sensations
- Music Time
- Matching Game
- Shape Sorter
- Fireworks
- Paint and Draw
- Trace-A-Shape

There are 3 ways to control the Mega Pod™:

1. Using the touchscreen. In this, your finger acts like a mouse. The cursor on screen should follow your finger when touching the screen. Touching the screen is like pressing the mouse button down.
2. Use the wireless keyboard and trackball unit (included).
3. Use a mouse (not included) or keyboard (not included) connected to the external USB port.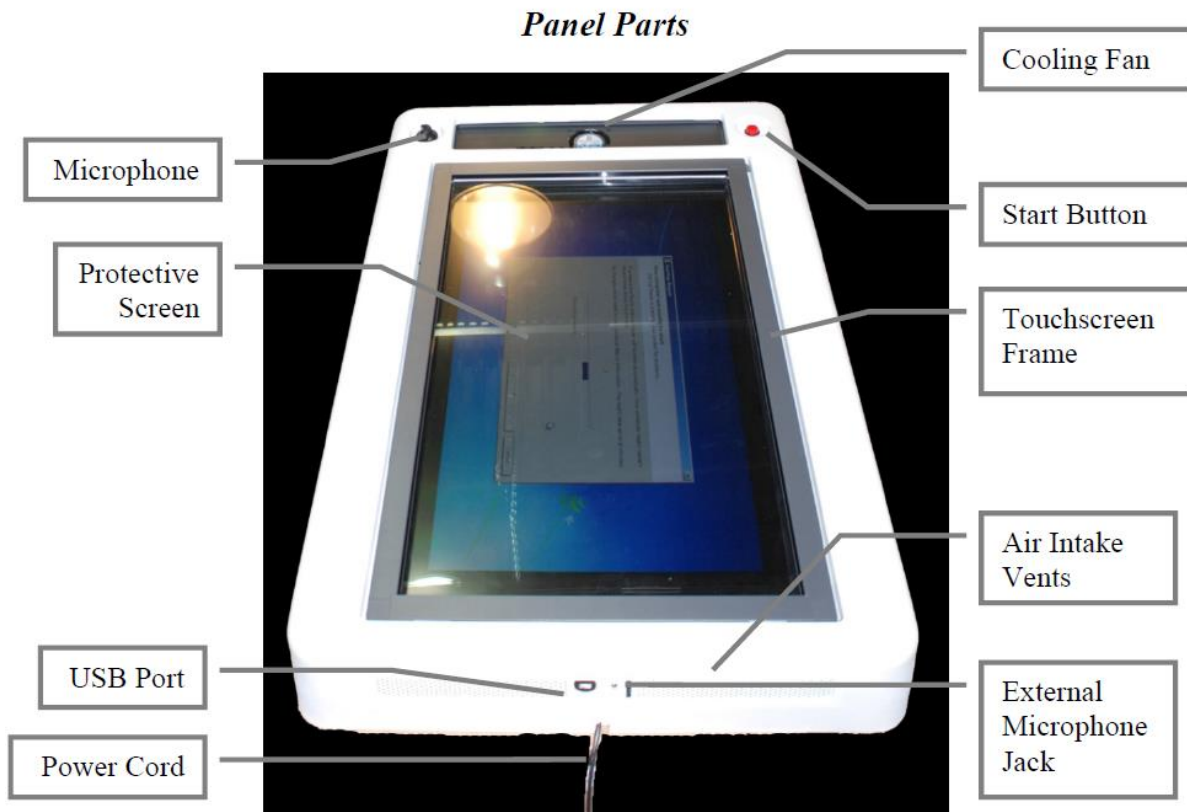

System Startup

Make sure that the panel is plugged into a socket that will always be powered (not switched).

Please note: NEVER disconnect power to the panel without using the Shutdown feature first (see below). Failure to do so may result in corruption of the data in the system.


1. Press and hold the start button for 3 to 5 seconds. After a few seconds the monitor should turn on and you will see the device booting up. This should take 1 to 2 minutes to complete.

2. After boot up is complete, you should see the calibration screen for the touchscreen. Follow the instructions on screen.
3. Once the calibration is complete, you will be taken to the index screen of the applications.

4. From the index screen, touch the activity that you want to play.

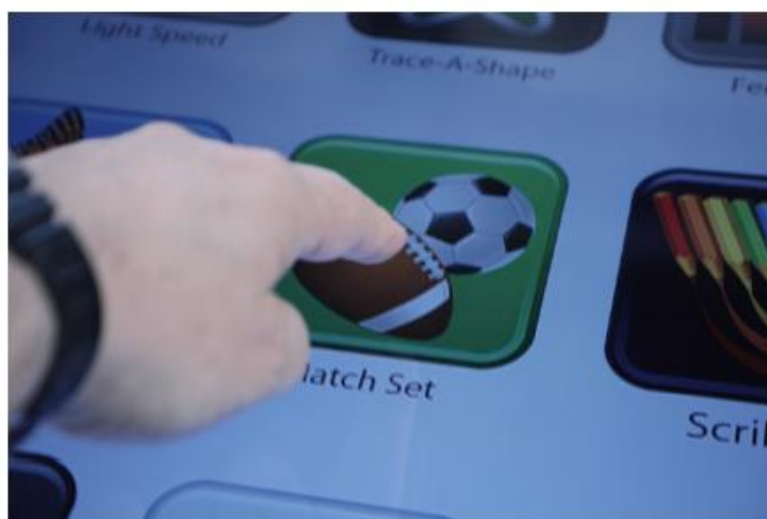Within the activity, you can choose of these options:

Return to the index screen.

Show the help for the activity.

Restart or reset the activity. (may not be present in all activities)

Adjust the volume of the activity.

Select a level of the activity. These may represent different levels of difficulty or simply different variations.

Other options specific to the activity may also be present.

Administrative Actions

- Go to the index screen.

Light Speed

Trace-A-Shape

Feedback

Kaleidoscope

Match Set

Scribble

Shapes in Space


Music Studio



Fireworks


- Choose the gear icon in the upper right corner. This will show a drop-down menu with the various administrative actions.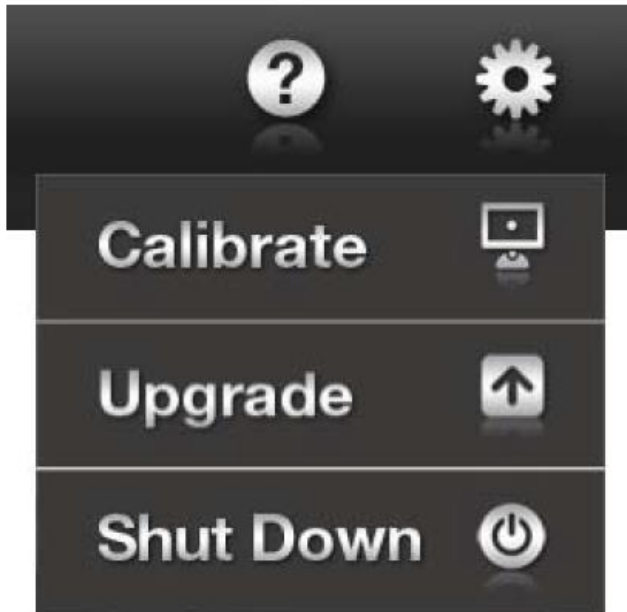

- *Calibrate* allows you to re-calibrate the touchscreen. This may be necessary from time to time if the Mega Pod™ has been left on for an extended period. If it becomes too out of calibration, you may need to use the wireless trackball to access the calibrate menu option.
- *Update* allows you to update the software on the panel. You will need to have a USB flash drive provided to you by ROMPA® for this. Please follow the instructions given to you on the USB flash drive.
- *Shutdown* allows you to turn off the panel. This must be done before disconnecting power from the unit.

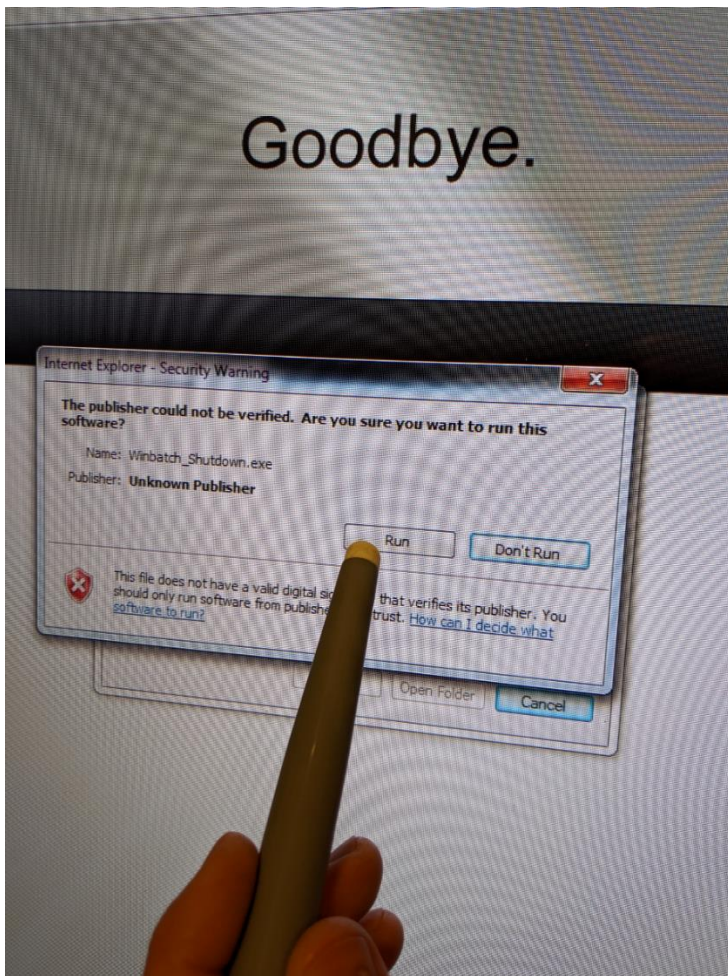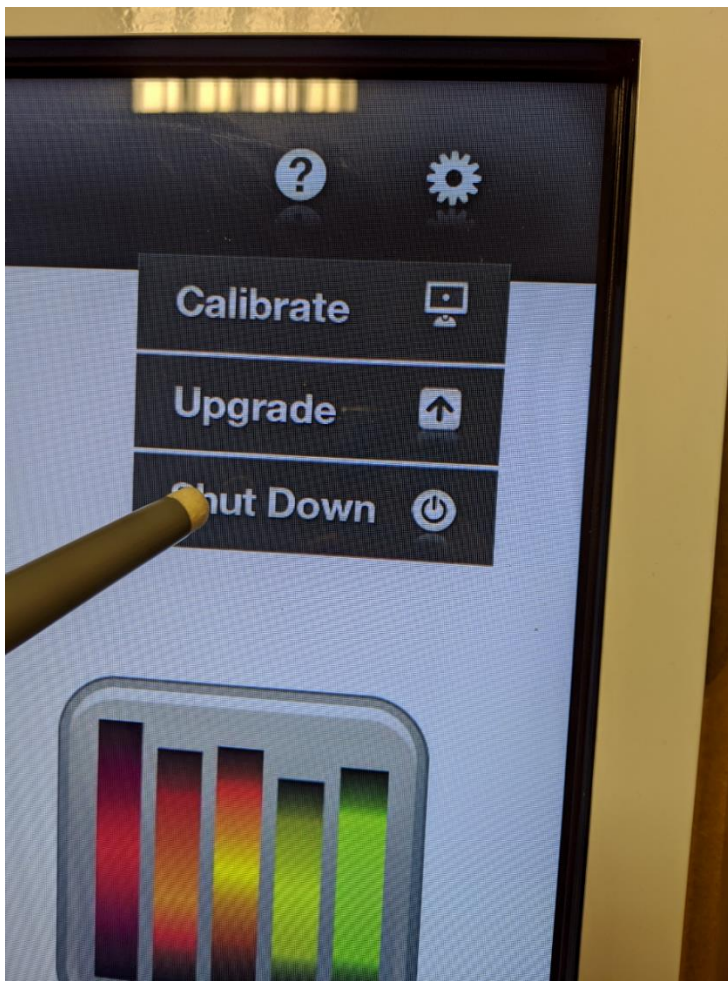Goodbye.

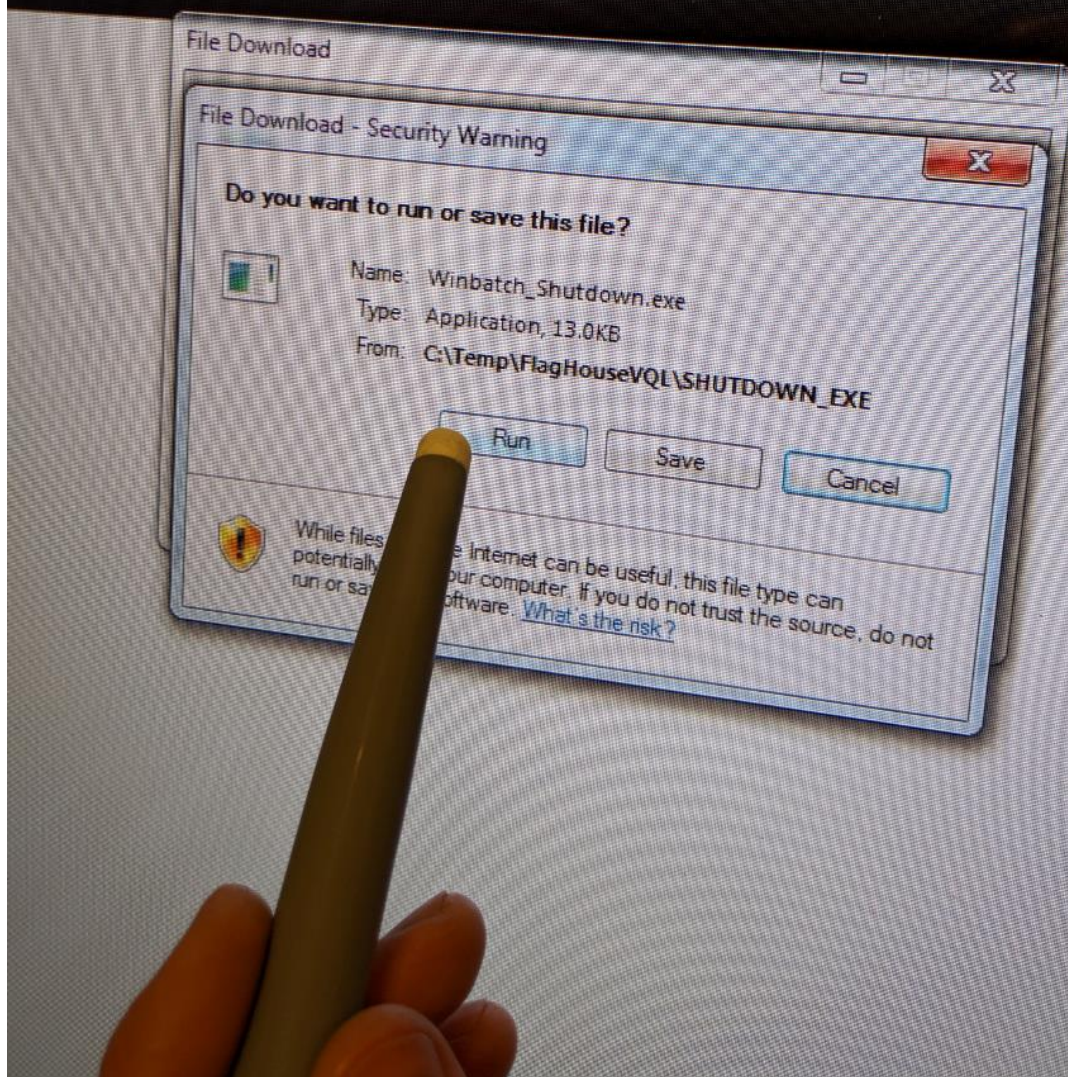11. Safety

- Always supervise the use of this product.
- Do not open up the panel when the panel is plugged in and switch off. Do not plug the panel in until the Mega Pod™ is fully assembled and installed.

12. Technical Specification

Size: Size 114 x 73 x 14cm

Weight: 32Kg

Power supply specification


Input: 90-240Vac 1.5-0.6A 50/60Hz (respectively)

Output: 24V DC 3.75A

Approvals: CE, UL CUL, FCC

13. Installation

- Choose a safe and suitable location for your panel. The panel is ONLY to be used in dry environments. The ambient temperature should not exceed 40°C.
- No fixings are included in this product for the wall. When fixing it to the wall please consult a suitably qualified professional builder or from an estates department to understand the construction of the wall the panel is to be fixed to, so that the correct fittings can be used
- Attach fixings such as mirror plates to the back of the panel. Check the fixings e.g. mirror plates you buy can easily support 32kg. You may wish to attach two mirror plates to the back of the panel at the top and two on the back at the bottom. When attaching fixings to the back of the panel please proceed with caution – be aware that there are electrical assemblies inside the panel.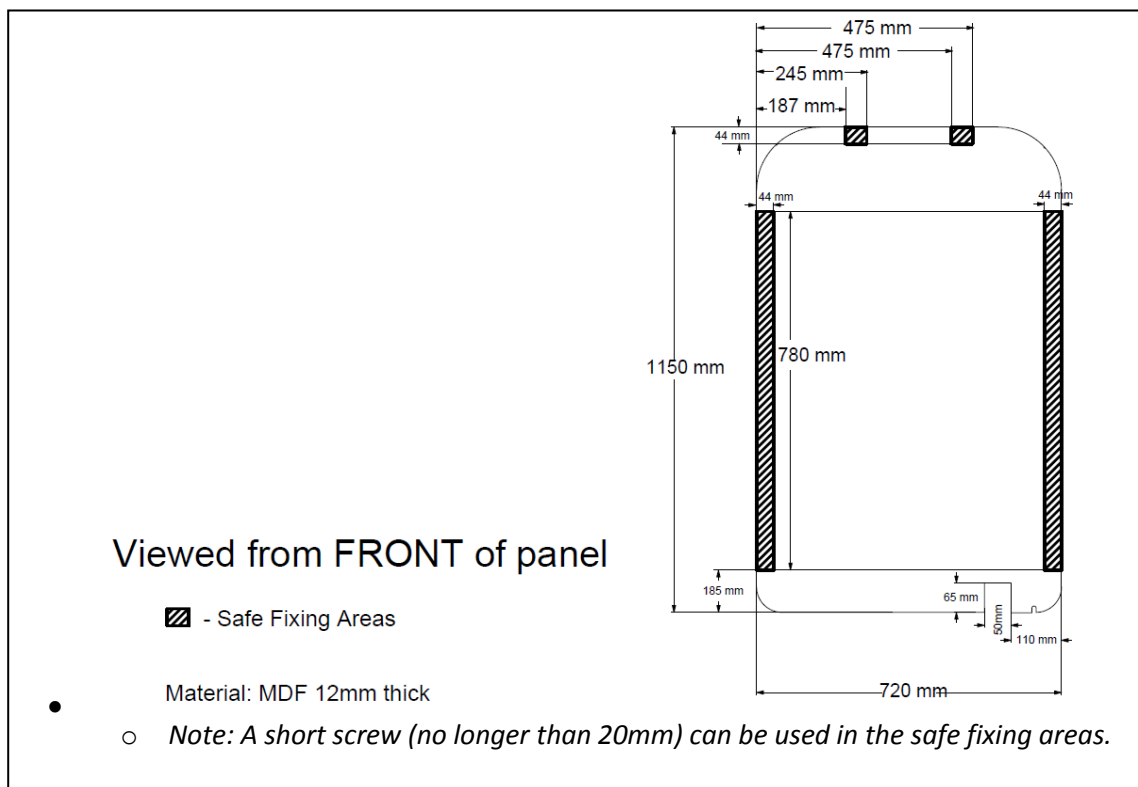

14. Care and Maintenance

- Wipe the exterior of the Panel with a clean, soft and very slightly dampened cloth. Do not allow the Panel to become wet.
- Do not strike the Panel – handle it with appropriate care.
- The Mega Pod™ does not contain any user-serviceable components. In the unlikely event of failure, please contact ROMPA®.

15. Troubleshooting

The screen cursor does not match your finger movements on screen

- Make sure that only one finger is within 1¼ cm (½”) of the touchscreen.
- If it is following, but not near your finger, the touchscreen may need to be re-calibrated (see *Administrative Actions* above).
- If the screen is not responding at all, the touchscreen may not be properly connected to the panel (see *Installation* above).

The Mega Pod™ orientation is landscape, not portrait


- If the picture appears sideways, it is set to landscape, not portrait. While using the keyboard in this setting, it will be easier to use the mouse by orientating the keyboard sideways like the current screen.
- Using the remote keyboard, while the Mega Pod™ is running, press the ‘flag’ key:


- This will bring up the Start Menu on the upper left corner of the screen.
- Left-click on the ‘Control Panel’ menu item.

- From the 'Control Panel' dialogue box that appears, left-click on the 'Adjust screen resolution'.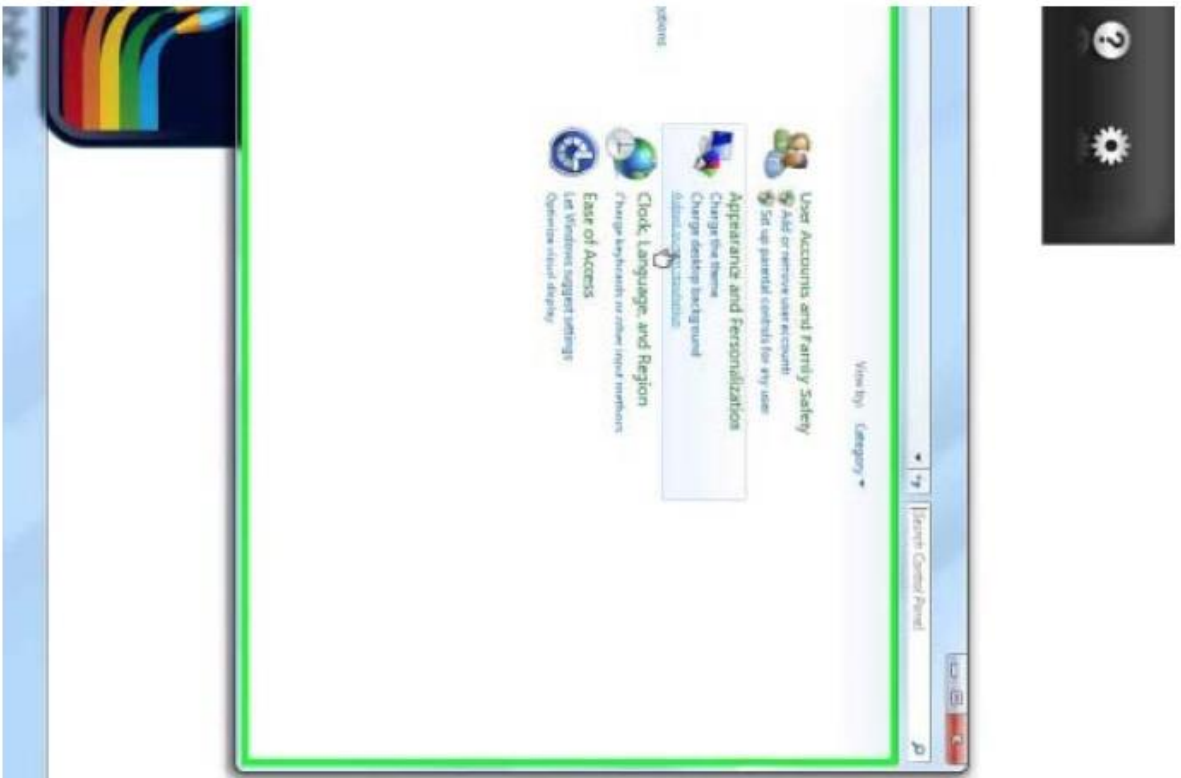


- In the 'Screen Resolution' menu under 'Orientation' left-click on the small arrow pull-down, select 'Portrait' and left-click.

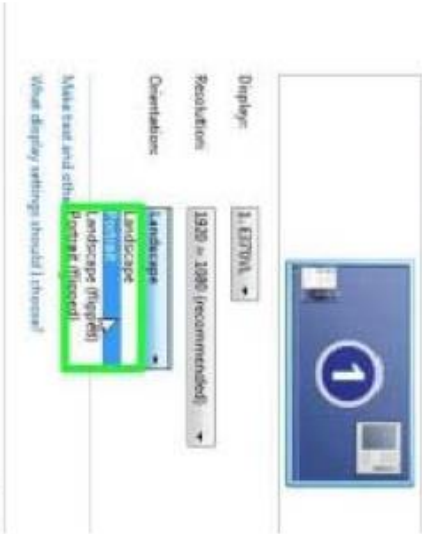
- Left-click on 'Apply'

- Left-click on 'Keep changes'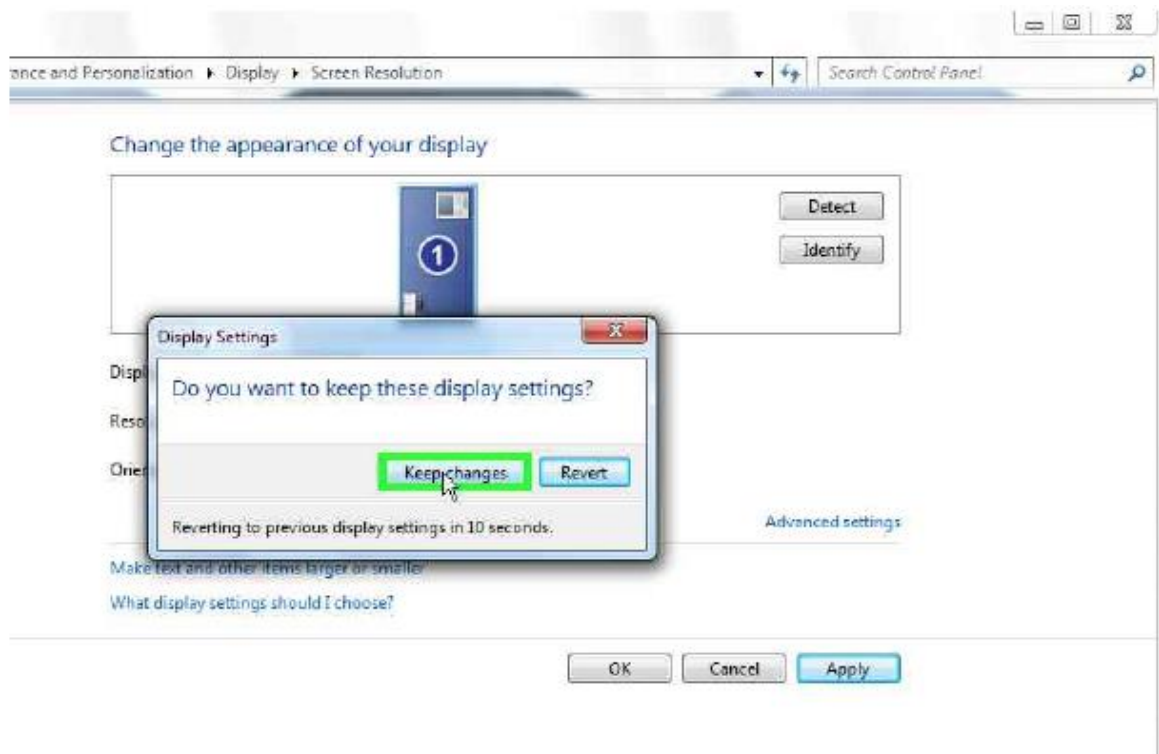

- Left-click on 'OK'

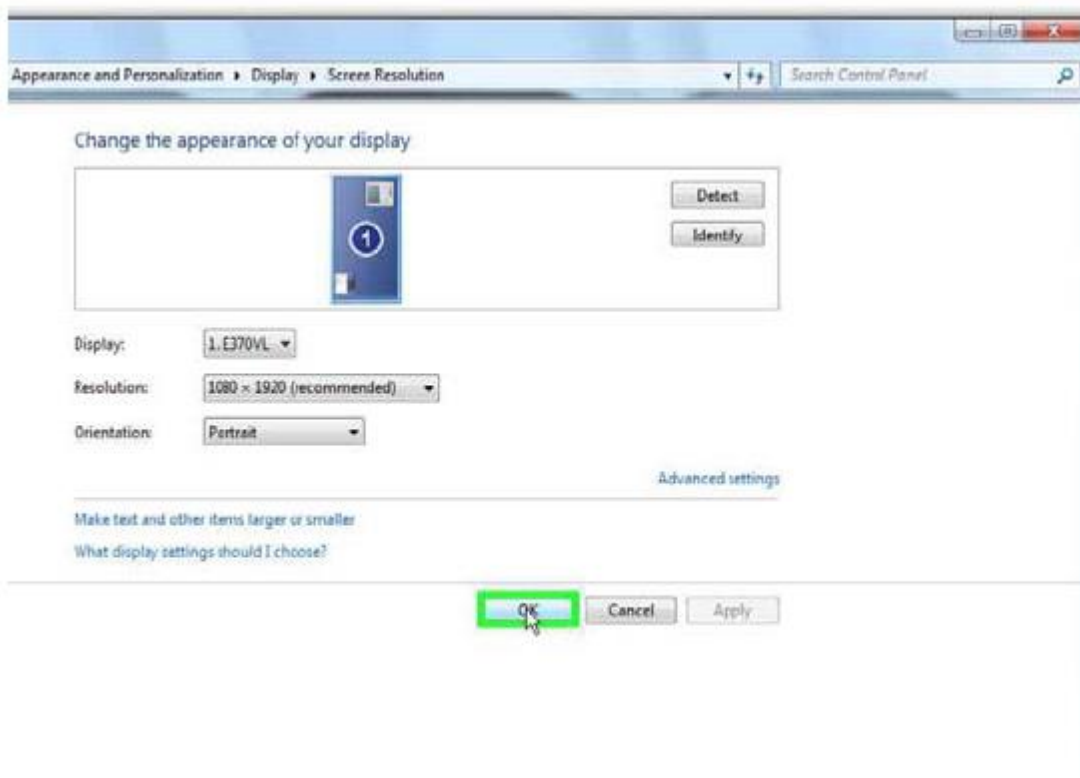- Left-click on the red 'X' in the upper right corner of the 'Control Panel' dialogue box to dismiss it and the procedure is complete.

- Left-click anywhere in the white space of the Mega Pod Menu to continue playing.

The volume needs adjustment

- Use the remote keyboard while the Mega Pod™ is running, to set the microphone input volume. Press the flag key:


- This will bring up the Start Menu on the lower left and the System Tray on the lower right. In the lower right-hand corner, hover the mouse pointer over the speaker icon and right-click.


- Select 'Recording devices' from the menu and left-click.


- From the 'Sound' dialogue box that appears, right-click on the 'Microphone' list item.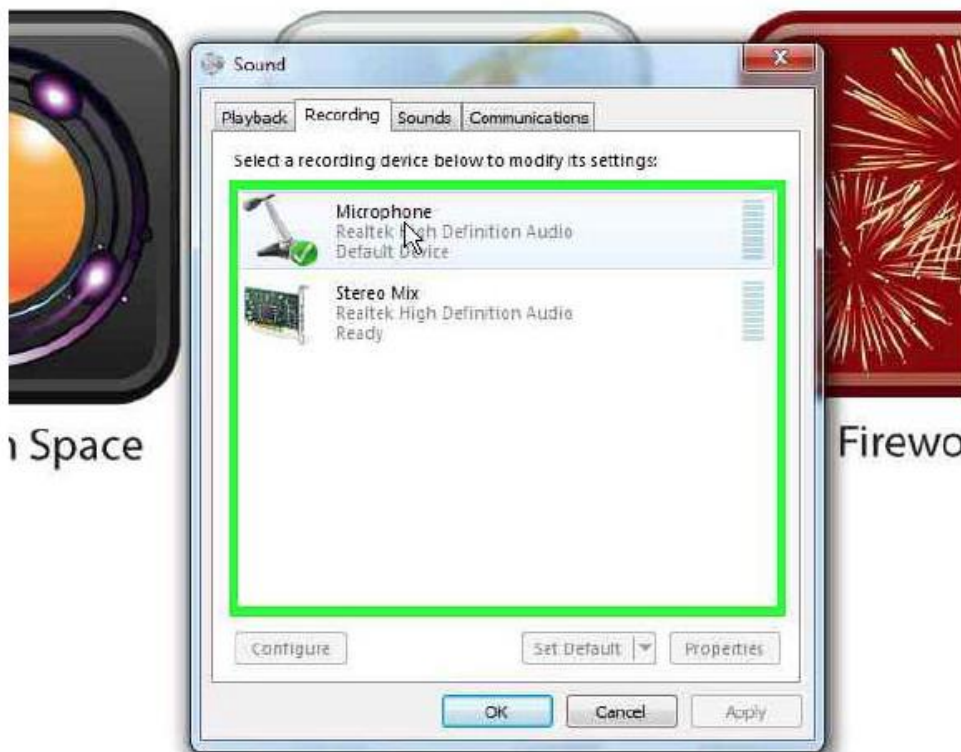



- Select the 'Properties' menu item and left-click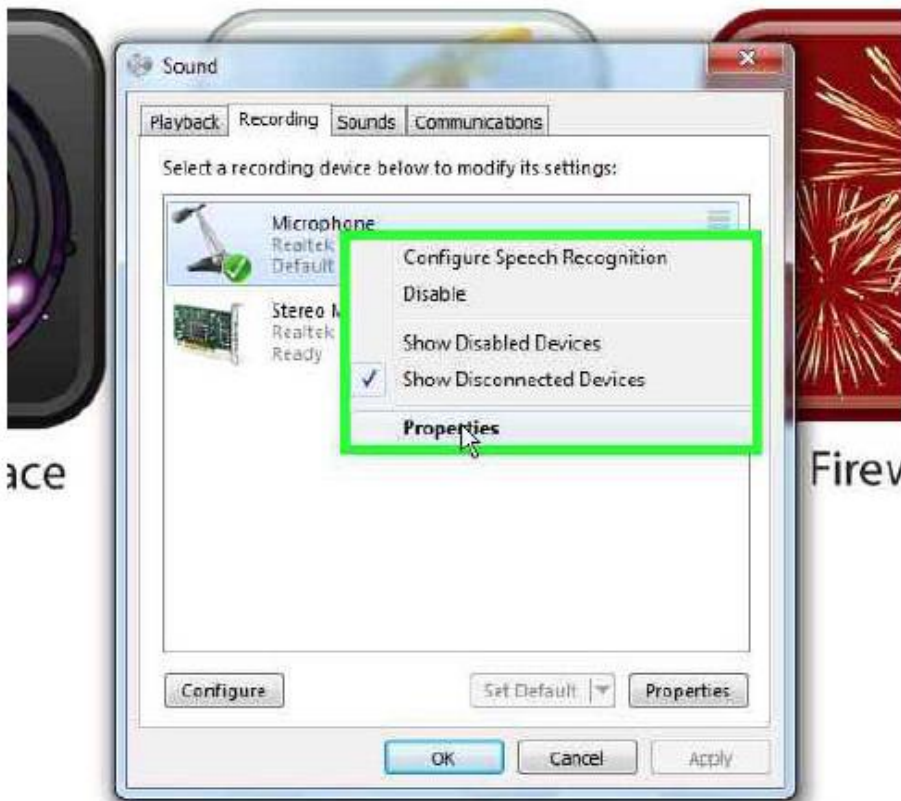


- From the 'Microphone Properties' dialogue box, left-click the 'Levels' tab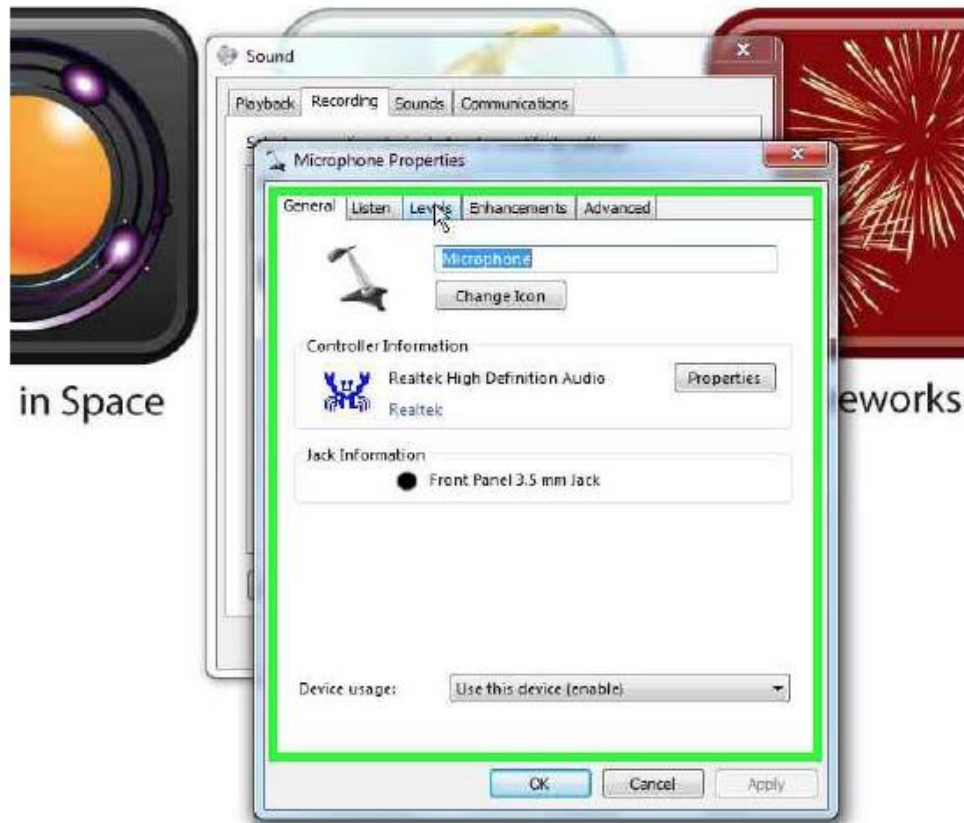

- To perform the next setting, hover the mouse pointer over the slider under 'Microphone'. While holding down the left-mouse key, use the roller ball to move the slider until the value is set to 15. Release the mouse key.
- Do the same for the slider under 'Microphone Boost', setting the value to +20.0 dB.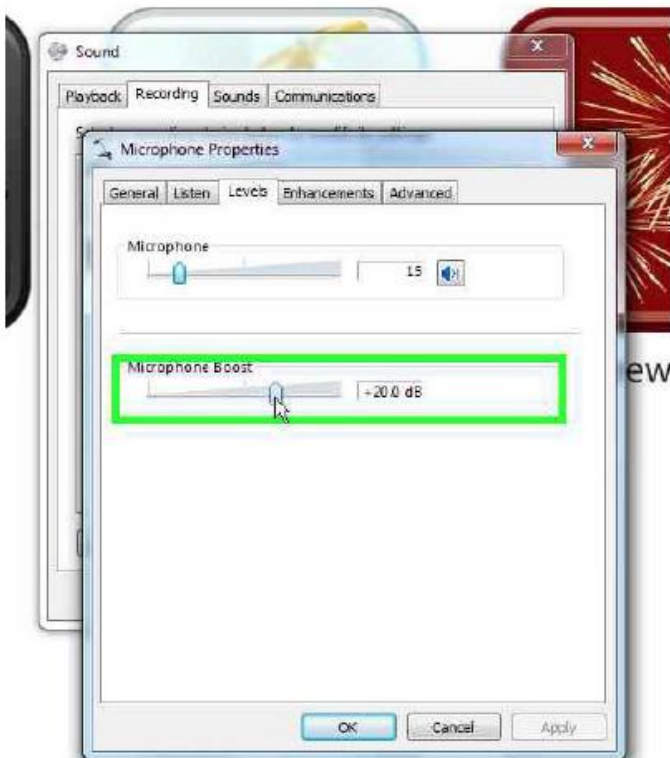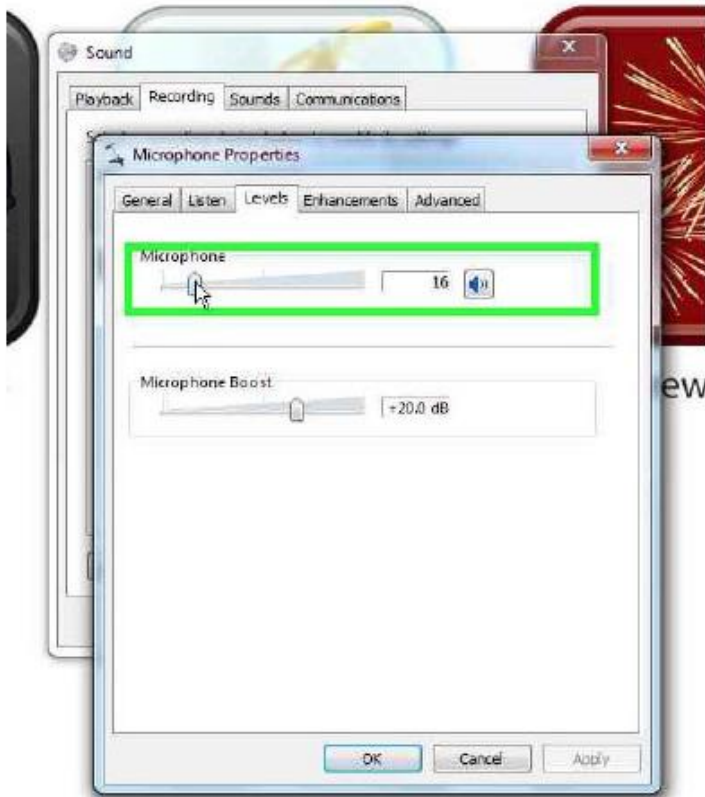

- Click 'OK' to dismiss the 'Microphone Properties' and 'Sound' dialogue boxes and the procedure is finished.

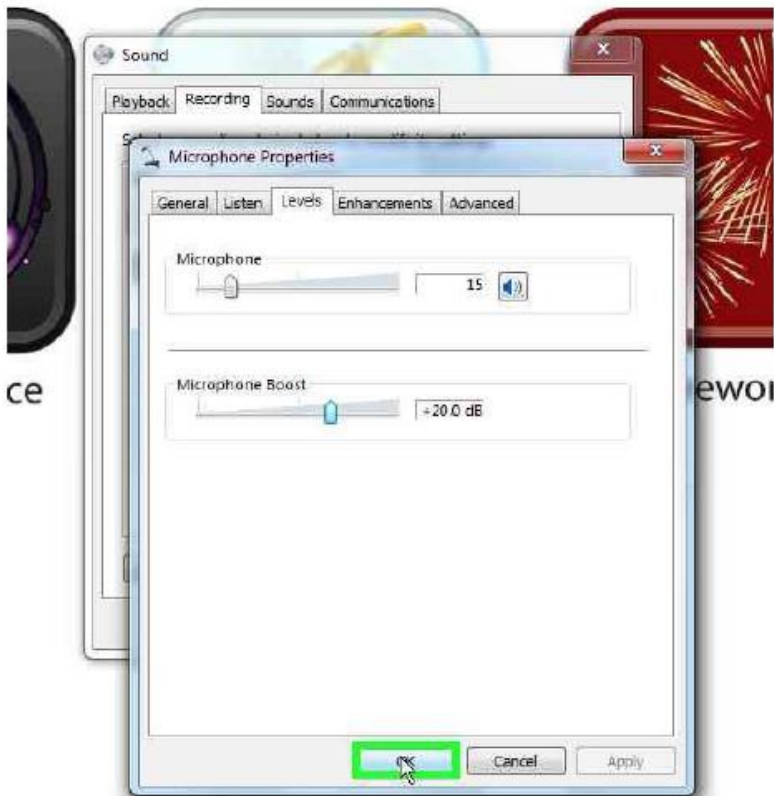- Left-click anywhere in the white space of the Mega Pod™ to continue playing.

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com

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