



### **1. Product Name**

Hanging Bracket

### **2. Product Code**

21928

### **3. Colour**

Blue

### **4. Brief Description**

Purpose made bracket designed for fixing to timber or metal. Fixings not included.

### **5. Contents**

- 1 x Hanging Bracket

### **6. Snoezelen® Stimulations**

- Vestibular

### **7. Best Use**

Use with Sensory Integration accessories and suspended equipment.

## **8. Compatible ROMPA® Products**

ROMPA® offers an extensive range of Sensory Integration products, including many swings that can be suspended from this Hanging Bracket. See our ever-expanding range at

[www.rompa.com](http://www.rompa.com)

## **9. Starting Up**

- Choose an appropriate location for the Hanging Bracket.
- Make a template using a piece of paper – trace the outline of the bracket and the holes. Hold this against your ceiling to mark the fixing points.
- Using appropriate fixtures, secure the Hanging Bracket in place.

## **10. Detailed Description**

Purpose made bracket designed for fixing to timber or metal. Fixings not included.

Note: Please ensure you ascertain from the appropriate authorities that the fixing point is suitable and will hold up to 450kg.

## **11. Safety**

- Always supervise the use of this product.
- Please ensure you ascertain from the appropriate authorities that the fixing point is suitable and will hold up to 450kg.


This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## **12. Technical Specification**

Size: 13cmW x 30.6cmL x 11cm  
10mm thick  
Weight: 3.5kg

### 13. Installation

1. Choose an appropriate location for the Hanging Bracket. Make sure the location is suitable and will hold up to 450kg.
2. As a guide as to the location of the holes to be drilled in your surface, we recommend that you make a template using a piece of paper – trace the outline of the bracket and the holes. Hold this against your ceiling to mark the fixing points. The diagram below is not to scale – it is purely an approximate illustration of the fixing points.

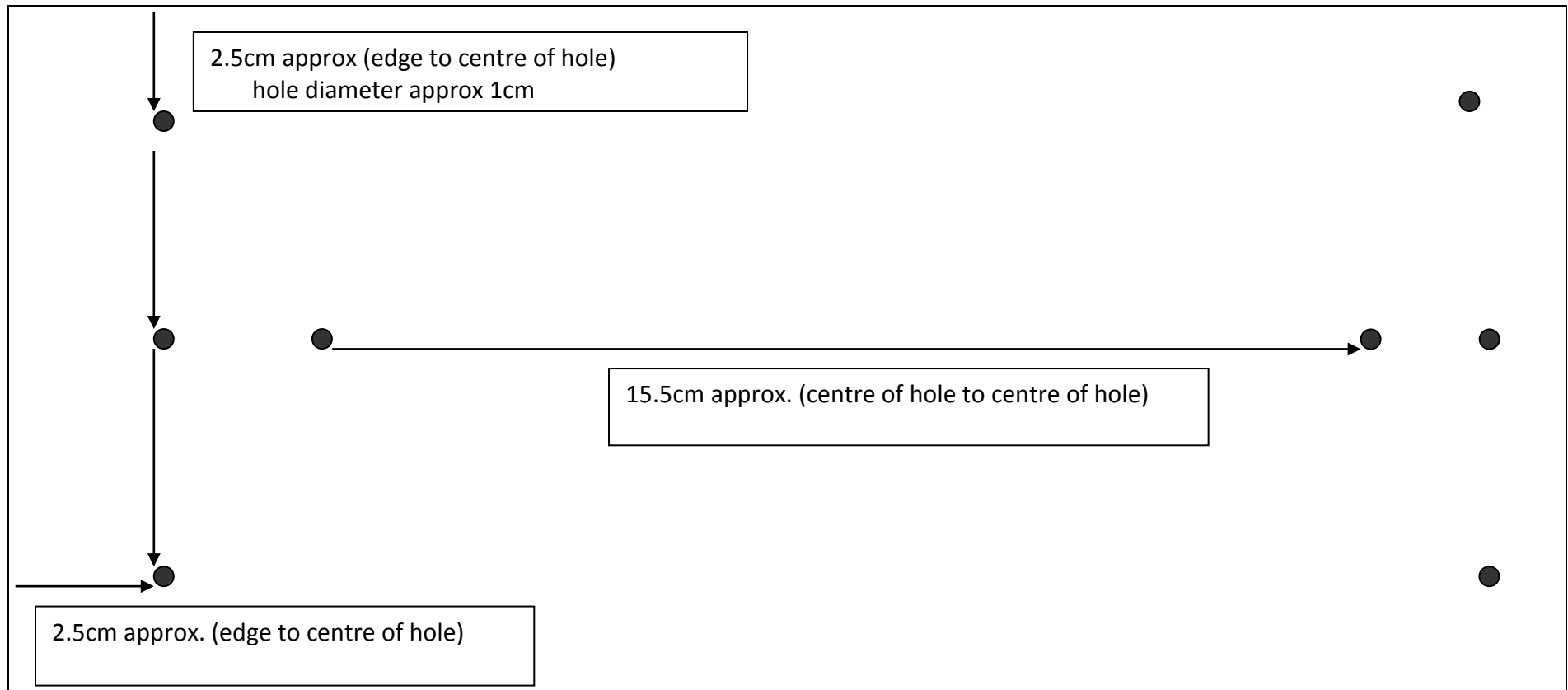

3. Using appropriate fixtures, secure the Hanging Bracket in place. Fixtures (e.g. nuts/bolts) are not included as these will depend on the construction of the surface the Hanging Bracket will be attached to. Please purchase appropriate fixings, and seek professional advice as required.
4. Use the Hanging Bracket with appropriate suspension equipment – Safety Snap, Height Adjuster, Safety Rotational Device etc. Thread your Safety Snap through the loop on the Hanging Bracket, and suspend other pieces of equipment from your Safety Snap, such as attaching a Rotational Device to the Safety Snap.

## **14. Care and Maintenance**

1. Regularly check the Hanging Bracket and maintain records of these checks. Check at least four times a year, especially if the Hanging Bracket is used often. Look for signs of excessive wear and tear. The bearing in the bracket allows movement to help to minimise wear, but given that the metal of the Safety Snap will be rubbing against the metal hook of the Hanging Bracket, signs of wear appear with frequent use. Replace the Hanging Bracket and Safety Snap as required.
2. Check that the Hanging Bracket is still securely in place.

## **15. Troubleshooting**

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)