



## 1. Product Name

Balanza®

## 2. Product Code

21932

## 3. Colour

As shown

## 4. Brief Description

A cleverly-designed balance board that can be used in a variety of ways.

## 5. Contents

1 x Balanza® with pump

## 6. Snoezelen® Stimulations

Movement

## 7. Best Use

Use this product on a flat, horizontal, stable and dry floor in an area free of obstacles.

## 8. Compatible ROMPA® Products

See [www.rompa.com](http://www.rompa.com) for similar products, as well as mats and helmets.

## 9. Starting Up

- Place on a flat stable surface.
- Adjust the inflation if required.

## 10. Detailed Description

A cleverly-designed balance board that can be used in a variety of ways. Stand on it with one or both legs; push up from it whilst holding the integral handles to exercise your arms or balance objects on it with a partner whilst seated next to it. Helps build co-ordination and balance skills. Made from a strong wooden base and an air-filled ball, which gives a 'floating' sensation and allows multi-directional movements. Maximum weight supported: 150kg.

Suggested uses (to be adapted according to your client's needs and abilities):



## **11. Safety**

### **Always supervise the use of this product**

On receipt, remove and dispose of all packaging responsibly.

Use this product on a flat, horizontal, stable and dry floor in an area free of obstacles.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

## **12. Technical Specification**

Weight: 2.25kg

Size: 55 x 35 x 7cm

Maximum weight supported: 150kg

Materials: phthalate-free and free from heavy metals  
High grade Ruton and birch wood ply (8 glued layers)

## **13. Installation**

1. Adjust the inflation using the supplied ball pump and the needle valve in the Balanza®. Add a little more air or release some air as required. Make sure that the product is at room temperature before making any adjustments. Always moisten the needle before inserting the needle into the valve. Do not inflate the ball beyond a diameter of 14cm (40cm perimeter approximately). The recommended diameter is 11 to 12cm.

## **14. Care and Maintenance**

- Regularly check the product for signs of wear and tear e.g. before each use. Replace the product if it is damaged.
- Store and use away from direct sunlight and away from sources of heat.
- Avoid contact with sharp or pointed objects. Remove shoes with buckles or metal fastenings before use. Wear sport shoes or soft shoes when using this product, or use whilst barefoot.
- This product is not intended to be thrown – handle this product with care.
- To clean this product use a soft clean cloth dampened with water or with a mild household cleaning detergent. Do not use solvents, dissolvers, acidic cleaning products or corrosive cleaning products. Keep this product dry.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at [producthelp@rompa.com](mailto:producthelp@rompa.com)