

**1. Product Name**

Airgo® Active Back Cushion

2. Product Code

21968

3. Colour

Black

4. Brief Description

Pliable air cushion to encourage comfortable sitting.

5. Contents

- 1 x Airgo® Active Back Cushion, already inflated
- 1 x pump

6. Snoezelen® Stimulations

Touch

7. Best Use

Use on a seat, such as a chair, aeroplane seat or car seat.

8. Compatible ROMPA® Products

21969 - Airgo® Active Seat Cushion



9. Starting Up

- Adjust the inflation.
- Place on a suitable seat and attach.

10. Detailed Description

Pliable air cushion to encourage comfortable sitting. May help support the back and promote correct posture. Made from materials that ensure optimal temperature and air circulation. Can be used with the Airgo® Active Seat Cushion. Black. Weight: 1.2kg. Maximum weight supported: 200kg.

11. Safety

Always supervise the use of this product

On receipt, remove and dispose of all packaging responsibly.
Use this product on a stable, indoor and dry seat.

This product is not a toy – it is intended for use under the supervision of trained professionals.


12. Technical Specification

Size: Size: 40 x 29cm
Maximum weight supported: 200kg
Materials: Free from phthalates and heavy metals




13. Installation

1. Adjust the inflation using the needle valve in the cushion and the included pump. The needle is recessed into the head of the pump handle (under the yellow labelled dot). The cushion is supplied inflated but may require adjustment according to your individual preferences and needs. Add a little more air or release some air as required. Make sure that the product is at room temperature before making any adjustments. Always moisten the needle before inserting the needle into the valve. Open the cushion zip. The valve is located at the back of the product. To deflate the cushion, insert the moistened needle and press out excess air. When inflating, pump in air, taking care not to inflate the cushion beyond the maximum firmness.

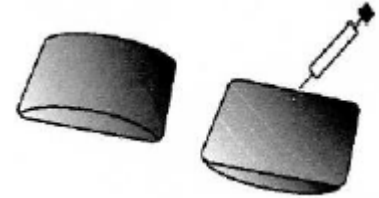

2. Use the belt to attach the cushion to your seat. Open the clip/Velcro® fastener (press the two tabs inwards of the clip to open and push these together to close till you hear a click). Place the flat side (the side with the zip) against the front of your seat. Make sure that the shorter belt is pointing towards the seat and the longer belt is towards the headrest.
3. Pull the shorter belt down and through the back. Pass the longer belt between the headrest and the back of the seat. The two belt sections can be connected with the clip/Velcro® at the back of your seat.
4. Make sure the longer belt has the length adjustment feature. Make sure the cushion is connected tightly to your seat.
5. Your shoulders should touch the back of the seat. This makes sure that the distance between your head and the headrest is correct.

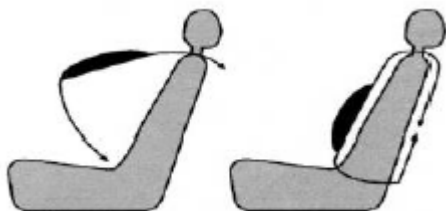

14. Care and Maintenance

- Regularly check the product for signs of wear and tear e.g. before each use. Replace the product if it is damaged.
- Store and use away from direct sunlight and away from sources of heat.
- Avoid contact with sharp or pointed objects.
- This product is not intended to be thrown – handle this product with care.
- The cover can be completely removed if required. If removing the cover, do so with care so as not to damage it as it is an intentionally tight fit to avoid creases and wrinkles. To clean this product use a soft clean cloth dampened with water or with a mild household cleaning detergent. Do not use solvents, dissolvers, acidic cleaning products or corrosive cleaning products. Keep this product dry.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com