



1. Product Name

Colour Changing Table

2. Product Code

22259

3. Colour

White exterior with colour changing light

4. Brief Description

A softly-lit table for sensory exploration. Hollow yet strong plastic. Aesthetically pleasing shape and colours. Especially effective in a darkened environment. Includes small remote control to choose from 16 colours or to select the fade mode.

5. Contents

- Colour Changing Table
- Mains power supply
- Remote control

6. Snoezelen® Stimulations

- Sight

7. **Best Use**

Use indoors or outdoors. If used outdoors, bring inside after use and store in a dry area.

8. **Compatible ROMPA® Products**

22813 Transparent Soft Bricks
22810 Transparent Linking People – Set of 36
22816 Exploration Tray
22811, 22812, 22814 Transparent Numbers
22817 Exploration Saver Pack

Visit www.rompa.com for the full range of sensory products that could be used with this table.

9. **Starting Up**

- Choose a suitable location where the item can be seen and interacted with safely.

10. **Detailed Description**

A softly-lit table for sensory exploration. Hollow yet strong plastic. Aesthetically pleasing shape and colours. Especially effective in a darkened environment. Includes small remote control to choose from 16 colours or to select the fade mode through a spectrum of shades. The colour spreads evenly through the whole shape. Includes a low voltage mains supply for charging the table. Once charged for 6 hours, the power supply can be detached and the table can be used where required – indoors or out - and illuminated for approximately 10 hours. The items shown are for illustration purposes and are not included. Water resistant and IP65 rated. Tested up to 80kg of load.

11. **Safety**



Please read all the following safety information before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Keep away from sources of heat and fire.
- Do not fully immerse in water, the table is water-resistant, not waterproof. The remote control and power adapter are not water-resistant and should be kept away from water.
- Do not leave outside, store in a dry area.

IMPORTANT: WARNING PHOTOSENSITIVE EPILEPSY

In fast flashing or strobe mode, the light source may potentially trigger seizures with children that have a history of photosensitive epilepsy. We therefore recommend that you do not use the fast flashing or strobe mode and suggest the remote control should only be used by a responsible adult or under adult supervision.

12. Technical Specification



- Size: 70cm diameter x 40cmH
- Tested up to 80kg of load
- Water-resistant and rainproof with an IP65 rating
- Rechargeable. Once charged for 6 hours, they can provide 10 hours of light
- Low voltage and energy efficient
- Long life LED bulbs
- Charging Time: 6 hours
- Remote control range: 3 metres
- Adapter Input AC100 – 240V 50/60Hz
- Operating current : 600MA

Power supply plug:



13. Installation

- Ensure that a risk assessment of the area has been made and the surrounding area is clear from obstruction and access routes are clear to enable safe use of the product.
- Select the adaptor for your territory, UK, US, EU or AUS and click securely into position. Push button to remove adaptor.
- Plug the power cable into the mains. The LED will light white while charging, then turn off when fully charged.
- The remote control can change the modes and colours of the shapes:

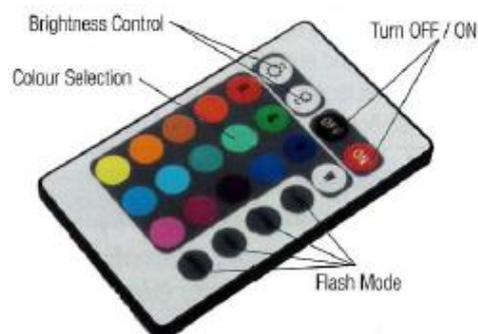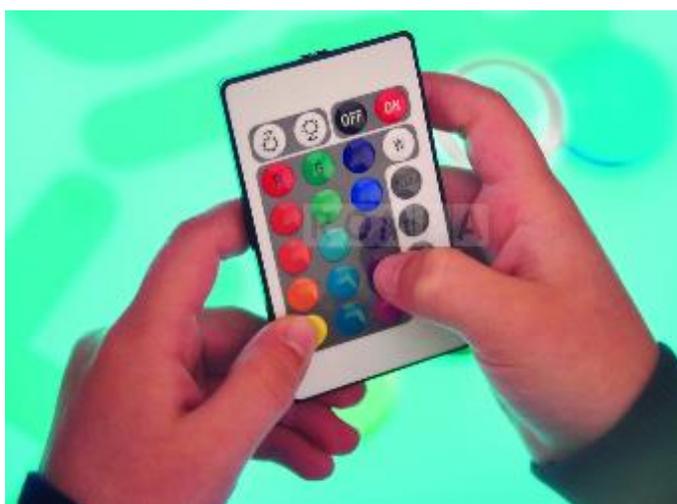

- **TURN OFF/ON:** Press to turn the LED light OFF/ON. There is also an OFF/ON switch on the base of each unit.
- **BRIGHTNESS CONTROL:** Adjust the brightness of the light up or down
- **COLOUR SELECTION:** 16 different colours can be select by pressing the appropriate button
- **FLASH MODE:** 4 different flash modes are available: Flash, Strobe, Fade and Smooth.

14. Care and Maintenance

- Extend battery life by charging when power is low, rather than allowing light to become completely exhausted.
- We advise cleaning this item with a non-abrasive mild soap solution and a clean cloth.
- The remote control and power adaptor are NOT waterproof. Please keep these away from water.

15. Troubleshooting

In the unlikely event of product failure, please check the following:

Problem: *The table does not light up using the remote control*

- Press the OFF/ON button on the panel on the underside of the table:
 - If the power supply is charging the table (if it is displaying the LED charging light), the problem is likely to lie with the remote control, rather than the power supply.
 - If the table lights up, keep pressing this OFF/ON button to click through the colours. Press and hold to switch off. If the table lights up using this manual button, check the battery in the remote control. If the remote control still does not work, even when a new battery is used, please contact us regarding a replacement remote control.
 - If the table does not light up when the OFF/ON button is pressed, please check that you are using the correct power supply (as shown in these instructions). Using a different power supply of a different/higher voltage is likely to cause a technical fault which will invalidate any possibility of repair/replacement. If the power supply is the same as that shown above, the table itself could be problematic and is likely to need returning to us for inspection.



Problem: *The table will not charge up*

- Ensure that the unit is switched OFF while charging. If the unit is switched ON, the power supply will act as a power source only and will not charge the unit. The unit needs to be switched off in order to hold charge.
- Check that the correct power supply unit is being used with the table. Only use the TickIT 5 volt PSU supplied with the unit.

Further copies of these instructions can be downloaded at www.rompa.com
We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com