



## **1. Product Name**

Jungle Bars

## **2. Product Code**

22266

## **3. Colour**


Likely to be as shown

## **4. Brief Description**

4 challenges in 1! Each 'bridge' has its own level of difficulty allowing grading and a variety of challenges to suit individual need. Exciting, inviting and enjoyable.

## 5. Contents

- 2 x wooden bars 250cmL (beech and plywood)
- 2 x wooden connection pieces (beech)
- 8 x wooden cross bars (beech and plywood)
- 4 x tread plates
- 1 x long bridge 250cmL (beech and plywood)
- 8 x adjustable end fasteners 200cm
- 4 x adjustable suspension ropes 150cmL (soft PP rope with a figure of 8)
- 4 x safety snaps
- Ropes for between the wood and suspension ropes (blue or green)

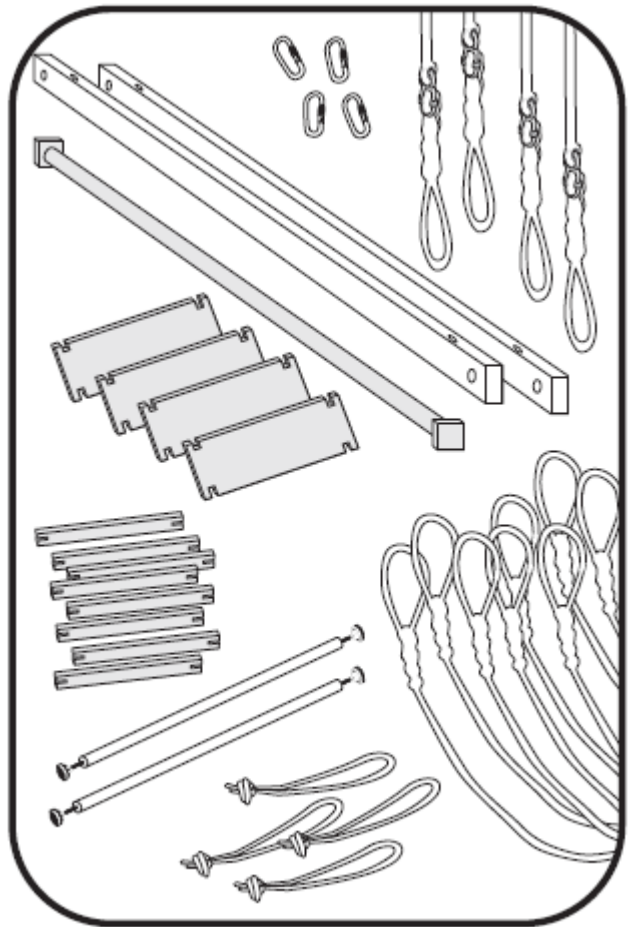

## 6. Snoezelen® Stimulations

- Vestibular
- Touch
- Proprioception

## 7. Best Use

For use by a suitably qualified professional, suspended from a safely-located, secure and appropriate suspension system. For indoor use only.

## 8. Compatible ROMPA® Products

Please note that a suspension system, Safety Rotational Device, Safety Snap, Height Adjustment System etc. are not included but are available from ROMPA®. ROMPA® offer a range of Sensory Integration suspension products such as swings, gliders, nets, bolsters etc. Please visit [www.rompa.com](http://www.rompa.com) for details about our full range.

- 21928 Hanging Bracket
- 21570 Safety Snap
- 21590 Locking Safety Snap
- 21548, 21607, 21546 Height Adjustment System
- 21559 Safety Rotational Device
- 21567 Vertical Bouncer Kit

## 9. Starting Up

There are no short-cuts to setting up Sensory Integration products. Read all instructions thoroughly before installing and using any swing.

## 10. Detailed Description

4 challenges in 1!

Each 'bridge' has its own level of difficulty allowing grading and a variety of challenges to suit individual need. Exciting, inviting and enjoyable. Wonderful for providing vestibular and proprioceptive input as well as fostering motor planning/skills, muscle-strengthening, spatial awareness, co-ordination and balance. Suspend from 2 points for a more dynamic and testing activity or from 4 points for more stable and controlled tasks. Maximum weight supported: 150kg

Size: 250cmL; adjustable suspension ropes 150cmL



## 11. Safety

### Always supervise the use of this product.

This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

- Do not exceed the 150kg (330lbs) maximum load.
- This equipment is intended for professional clinical use only under supervised conditions by a trained specialist. Use without proper supervision or knowledge is potentially dangerous and should not be undertaken or permitted. Before using these products, make sure you know your limitations and that of your equipment.
- Client safety is dependent on the appropriate choice of activity to match, yet challenge physical ability.
- Make sure that your client is appropriately dressed – avoid any loose clothing that could become entangled in the equipment. Remove earrings, necklaces etc. before use. Consider appropriate footwear – avoid shoes with buckles, dark soles etc. which could damage the swing. Stocking feet (socks) are best. Plimssoles or soft-soled shoes may also be worn.
- The layout of the activity area, proper installation and maintenance of equipment is critical – use this swing appropriately and safely.
- Do not use or store outside.
- Take time to familiarize yourself with the use and maintenance of this equipment before using and file these instructions in an accessible location. These instructions must be read and be familiar to anyone who is using this equipment.
- All parts of the Safety Snap should be thoroughly checked at least on a monthly basis and before use along with all suspended equipment including the Suspension System.
- Check all points where movement occurs for wear of any kind. All wear points should be checked regularly and recorded on the maintenance checklist enclosed to monitor their condition. Worn parts should be replaced or repaired immediately.
- Floor mats of an appropriate thickness should be positioned properly underneath and around all suspended equipment. Wall matting/cushioning should also be considered if appropriate.
- Steel safety snaps are recommended as aluminium Safety Snaps wear more quickly.
- The suspension point must have a clear distance of at least 6 feet from walls and other obstacles.
- Risk assess before use. Inspect equipment for wear and tear before use and discontinue use if any doubt.
- Your client may require assistance to enter and exit the swing. Assess how to transfer safely before attempting use.



## 12. Technical Specification

Size: 250cmL; adjustable suspension ropes 150cmL  
Maximum load: 150kg (330lbs)  
This product meets the requirements of 93/42/EWG Medical Devices



## 13. Installation

- Please ensure that you ascertain from the appropriate authorities that the fixing point is suitable and will hold more than the 150kg maximum load (allow for movement for a working load) at up to a 45° angle in any direction.
  - Please check equipment immediately for any damage that may have resulted during shipping. If any damage has occurred, please contact ROMPA® immediately.
1. Make sure the swing is set up safely and securely before use.
  2. You may wish to use a Height Adjustment System (not included but available from ROMPA®).
  3. A Safety Rotational Device must be used for all activities, not just for rotation, as this helps prevent wear on the suspension point (not included but available from ROMPA®).
  4. All suspended equipment requires a minimum ceiling height of 2.5 metres/8 feet.
  5. Ensure that suspended equipment is clear of other obstacles
  6. A clearance of 1.8 metres/6 feet around the hanging item must be observed
  7. The distance between the floor and the product should be 1'.
  8. Make sure that all adjustments are secure and tight before use.
  9. Check all materials and connections for wear before using.


Step 1

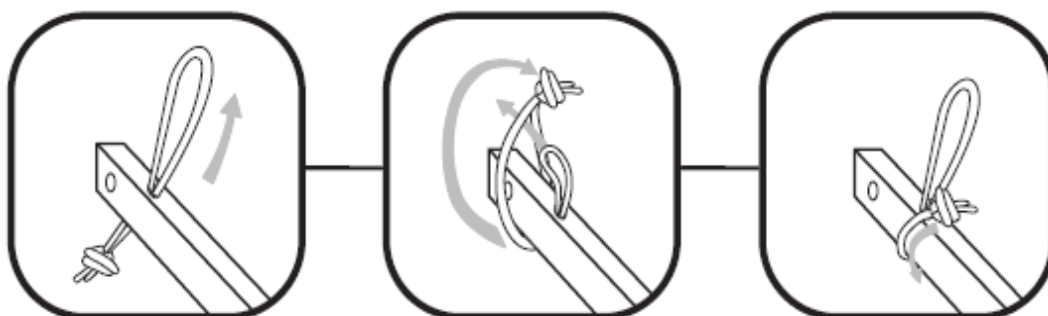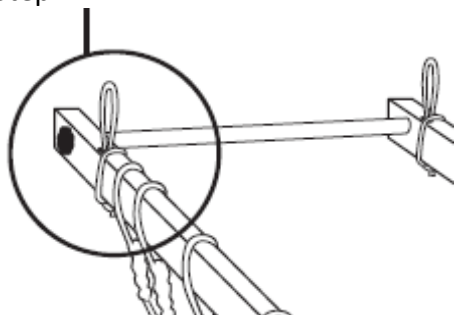

Step 2

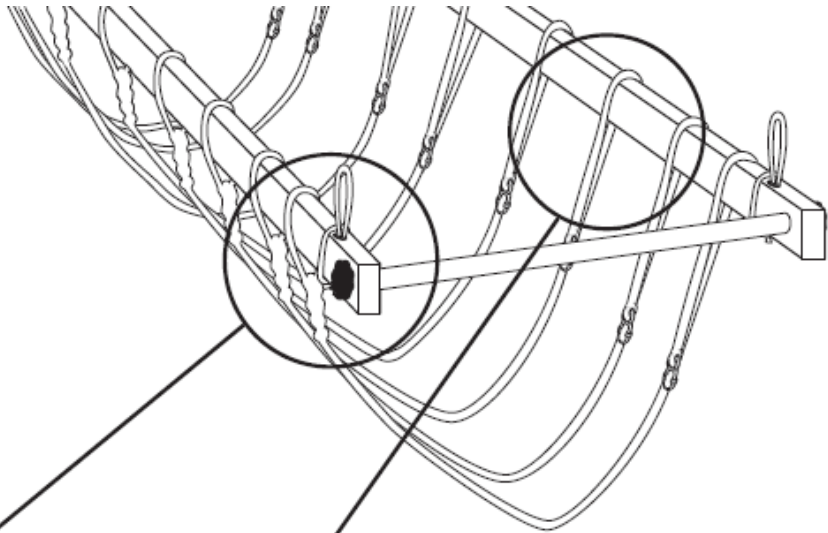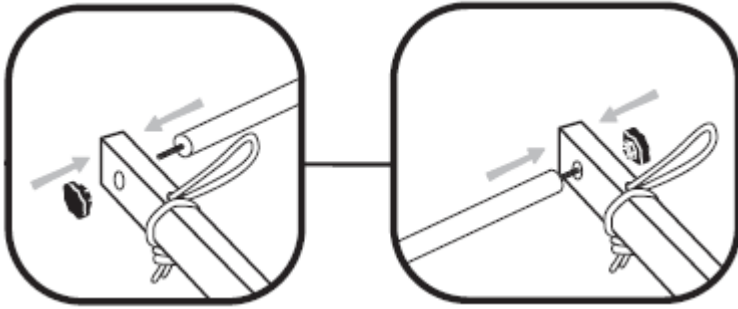

Step 3

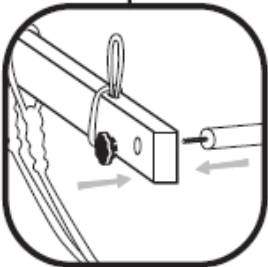


Step 4




To adjust the height



Step 5

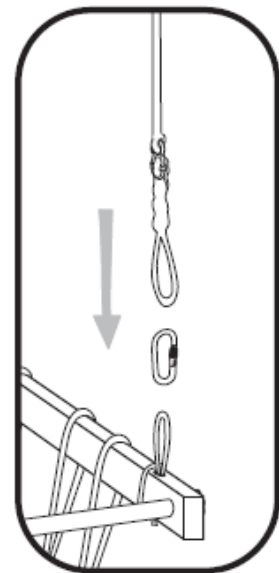

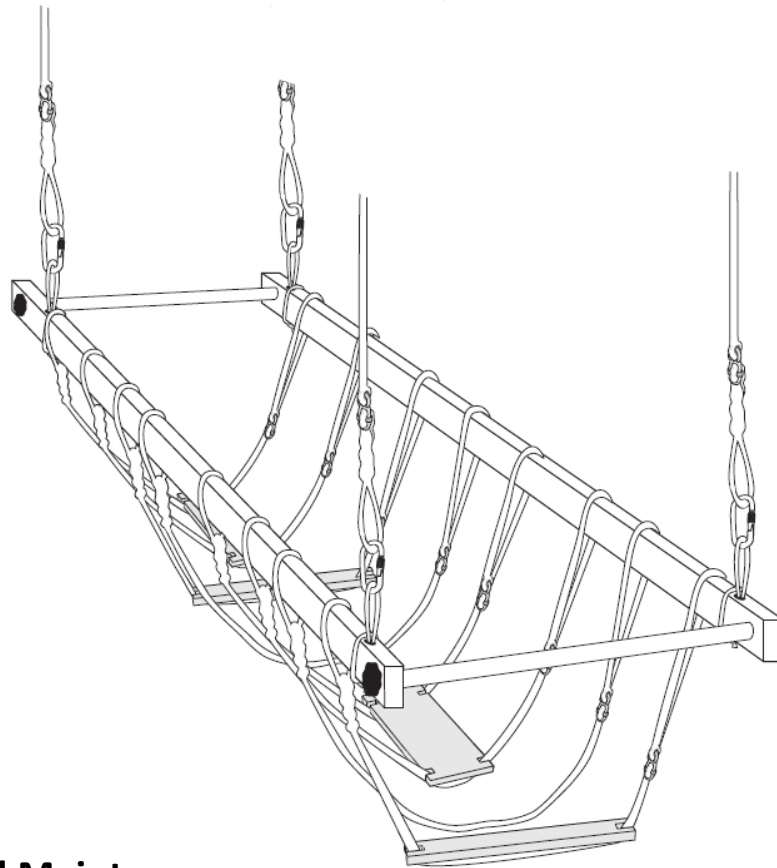

## 14. Care and Maintenance

1. Wood is a natural product and therefore wipe the wooden parts of this product with a soft, slightly damp and lint-free cloth. If you wish to use a cleaning product that is intended for wood, spot-test the wood in an area that won't be especially visible first.
2. A solution of edible oil and vinegar can be applied in small amounts to the wood to preserve the visual appeal of the wood. Apply with a soft, lint-free cloth.
3. Ropes may be cleaned with a water and mild soap solution.
4. Should you need to use a disinfectant product, use only alcohol-based disinfectants (Ethanol/Propanol).
5. Avoid harsh and abrasive cleaning products and methods. Do not use solvents.
6. Use and store in a dry environment.

## 15. Troubleshooting

*Not applicable*

Further copies of these instructions can be downloaded at [www.rompa.com](http://www.rompa.com)

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

[producthelp@rompa.com](mailto:producthelp@rompa.com)

