



1. Product Name

Floating Cloth

2. Product Code

22480

3. Colour

As shown

4. Brief Description

A unique Sensory Integration resource for providing tactile, vestibular and proprioceptive input. Encourages motor planning and co-ordination.

5. Contents

- 1 x cloth with 4 x spring ropes
- 8 x safety snaps
- 4 x covered spring-loaded fixings

6. Snoezelen® Stimulations

- Vestibular

7. Best Use

Use from a safe and appropriate suspension system.
Suspend from 2 or 4 points



8. Compatible ROMPA® Products

21928 – Hanging Bracket

21570 – Safety Snap

See www.rompa.com for mats and a large range of Sensory Integration products

9. Starting Up

- There are no short-cuts when setting up and using Sensory Integration equipment. Please keep these instructions in a safe place and refer to them before each use.

10. Detailed Description

A unique Sensory Integration resource for providing tactile, vestibular and proprioceptive input. Encourages motor planning and co-ordination. Can also help build muscle-tone. For balance and therapy or for relaxation. Can be used outdoors (and then stored indoors). With four covered mounting springs for dynamic movement. For suspension from four points or two (for use as an enclosure). Includes cloth, springs with mounts and 8 safety snaps. Washable polyamide with acrylic straps. Intended for use by an adult or a child.

Maximum load: 200kg.

Size: 250 x 250cm

11. Safety

- Always supervise the use of this product
- Make sure that this product is properly installed before use.
- Do not exceed the maximum load of 200kg.
- Sensory Integration products are intended to be used by professionals.
- Use safety mats on the floor, as well as on the walls as necessary.
- Use a safety helmet, and any other appropriate safety measures if required for your client's needs.
- Use only steel safety snaps as aluminium safety snaps wear more quickly.
- For indoor use only.
- Check all hardware and stitching before each use to check these are intact.
- Use only the components supplied (do not attempt to swap any of these for alternative parts).
- Make sure the client is wearing appropriate clothing (e.g. sportswear) and that jewelry is removed.
- If in any doubt about the appropriateness of Sensory Integration therapy with your client, seek advice from a medical professional (e.g. doctor).


This product is not a toy. Any activity involving motion, height, or rotation may cause serious injury. This product is intended for use under the supervision of trained professionals.

12. Technical Specification

Maximum load:	200kg
Size:	cloth 250 x 250cm; polypropylene cords with tabs 250cm
Fabric:	Cordura (polyamide)
Weight:	4.5kg (without accessories)
Colour:	likely to be dark blue and red
CE:	meets requirements of 93/42/EWG

13. Installation

- Allow plenty of unobstructed space around your swing. Suspended equipment should be positioned away from walls and obstacles.
- Install all points equally and in the same way.
- The Floating Cloth can be fixed to a wall- or ceiling- suspension system, from 2 or 4 points.
- This product may be installed indoors or outdoors. If used outdoors, store indoors after use.

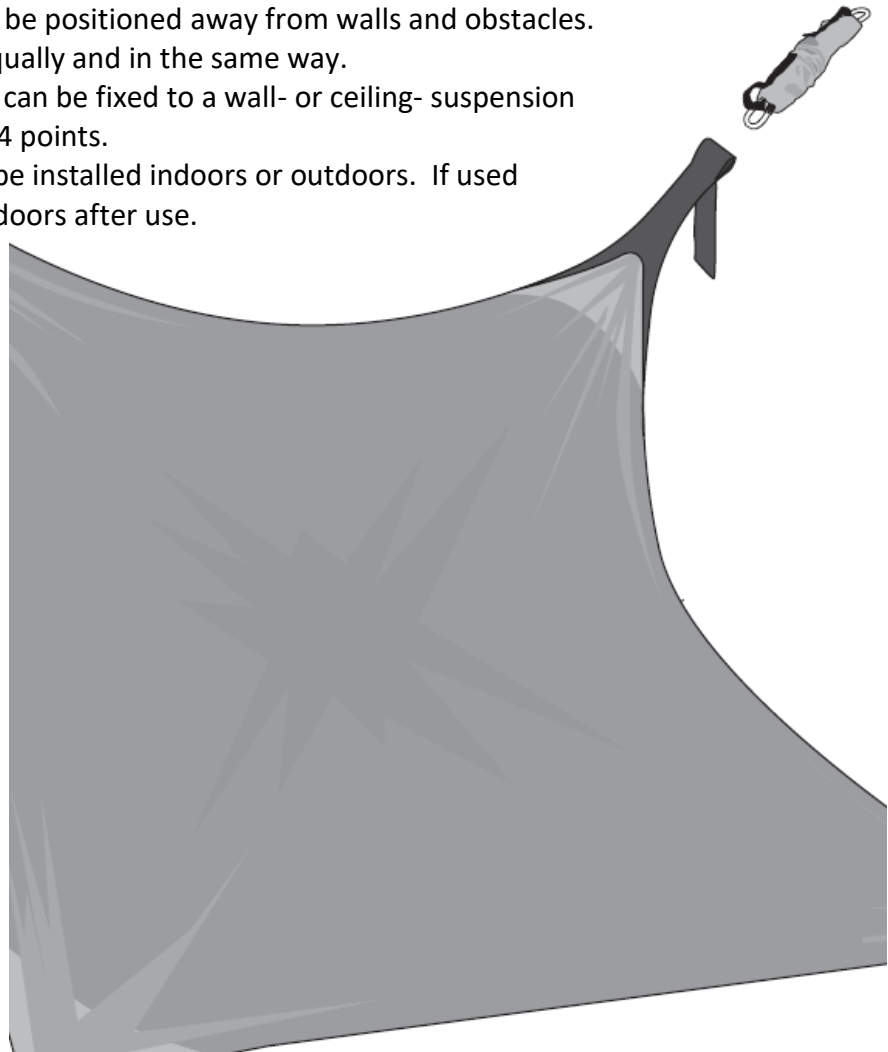

14. Care and Maintenance

1. All the parts of your suspended products should be checked on a regular basis (at least monthly, and more if the swing is used often). Use a maintenance sheet (such as the example one provided) to record these checks.
2. Check all the parts for signs of wear and tear before each use and replace these as appropriate. For example, check the cloth for loose stitching.
3. Use and store this product indoors, in a safe place.
4. Dust or wipe the components clean with a soft cloth or soft brush. For dirt that can be removed this way, hand-wash the fabric with water and a mild soap solution. Do not machine-wash as this could damage the integrity of the fabric. If disinfectant is necessary, only use alcohol-based disinfectants such as ethanol or propanol. Do not use any other kind of cleaning products as these are likely to irrevocably damage the fabric. Do not use solvents. Allow to dry completely before use. Do not fold the fabric until completely dry to avoid dye-transfer.
5. Check that the product is in a clean condition and in good working order before each use.

15. Troubleshooting

Not applicable

Further copies of these instructions can be downloaded at www.rompa.com

We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at

producthelp@rompa.com

