



1. Product Name

Wi Fi Bumpas – Set of 8 by ROMPA®

2. Product Code

22716

3. Colour

As shown

4. Brief Description

Press a coloured circle to change the colour of light in a ROMPA® Wi Fi receiving product.

5. Contents

- 1 x wall board with 8 bumpas attached
ROMPA® Wi Fi receiving products are not included

6. Snoezelen® Stimulations

- Visual

7. Best Use

Wall mount in a darkened room with ROMPA® Wi Fi products in it.

8. Compatible ROMPA® Products

ROMPA® offers a large range of Wi Fi responsive products - the range is continually growing – please see our catalogue and website – www.rompa.com. Some compatible products include:

21098, 20848 Sensory Magic®
19771 Midi Bubble Tube
19772-5 Maxi Bubble Tube
19777-80 Aquatic Bubble Tube
19776 Giant Maxi Bubble Tube
19767-770 Ball-Bubble Tube
19210 Light Source
21452 Two Port Light Source
19214 Colour Changing Panel
19209 Colour Match Panel
18523 Bubble Wall Panel
19603 Catherine Wheel Panel
18537 Interactive Fibre Optic Fountain Panel
18544 Fireworks Panel
20132 Snow Panel
19723 Light Diffraction Totem
20566 Infinity & Beyond
20287 Wi Fi Starry Sky Panel

We also manufacture made to measure cushioning. Visit www.rompa.com for more information.

9. Starting Up

- Choose a suitable location indoors.
- Plug in the board into an electrical wall socket.

10. Detailed Description

Press a coloured circle to change the colour of light in a ROMPA® Wi Fi receiving product. Helps encourage awareness of cause and effect and gross motor skills. The colours left to right are: red, green, blue, yellow, orange, pink, purple and white.

11. Safety

- Use indoors, in dry environments.



Please read all the following safety information before each use

ALWAYS SUPERVISE THE USE OF THIS PRODUCT

- Remove all packaging and either store this in a safe place out of reach of clients or dispose of this responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- The Bumpas contain a foam filling. Do not attempt to access the interior of this product. Never allow children or vulnerable people to access the interior part of this product.
- Keep away from sources of heat and fire.

12. Technical Specification

Size: approximately 2.4mW
Tallest bumpas in centre: 145cmH
Middle bumpas: 135cmH
End bumpas: 125cmH

Power supply: 12vDC 1500mA

13. Installation

1. Choose an appropriate location for this product bearing in mind the following:
 - 1 wall socket is required
 - The users need to be able to reach the coloured circles of the Bumpas
 - The users should be able to see the Bumpas and the receiving Wi Fi product(s) in the room in order to appreciate that pressing a coloured circle is activating the corresponding coloured light in the room.
2. Plug in the cable from the power supply housed in the back of the board of bumpas into a wall socket. The cable comes out of the board at the bottom left of the back of the board.

OPERATING INSTRUCTIONS

When pressure is applied to a coloured circle the corresponding colour will be activated on the ROMPA® Wi Fi receiving product(s) in the room e.g. light source, colour wall controller etc. A firm press is necessary on the circle. The colour will not be activated unless the circle is pressed – pressing anywhere else on the bumpa will not activate the corresponding coloured light.

The light cast by the colour wall controller will remain on the selected colour until someone presses another circle. Should no-one press another circle, after one minute, the colour wall controller will revert to its automatic cycle – the light will scroll through a sequence of colours. In other words, the bumpas have a ‘latched’ operation – the colour stays on for one minute unless another circle is activated.

14. Care and Maintenance

Fabric Care

- Always supervise the use of the product.
- Organise a regular cleaning schedule.
- Vinyl will crack, tear and rub off if subjected to sharp objects or continual rubbing by hard objects. Remove shoes, and avoid buckles, zips or sharp/hard/pointed objects coming into contact with the surface.
- Check seams regularly to ensure that these are not worn or split and the interior filling cannot be accessed.
- Wipe off accidental marks/liquids at the time they occur or are noticed.
- From time to time, if possible, lift floor cushion or move furniture to clean all surfaces and the floor underneath.
- Avoid eating and drinking whilst using these products.
- Steam cleaning is not recommended.
- Do not place near sources of heat or fire that may damage the surface (radiators, fires, underfloor heating, hot central heating pipes etc.)
- Direct contact with bright sunlight may affect material over time.
- Do not saturate the material with lots of water.

PVC (Vinyl)

- Use a soft brush remove all loose dust/dirt
- Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.

15. Troubleshooting

In the unlikely event of product failure, check:

- Is the product plugged in?
- Are there ROMPA® Wi Fi receiving products in the room? If so, are these plugged in and switched on?

Further copies of these instructions can be downloaded at www.rompa.com
We hope you find our instructions invaluable. If you have any suggestions for improving them further your comments will be greatly received – please contact us at producthelp@rompa.com