




Life Changing Sensory Solutions For All

**snoezelen®**

# INSTRUCTION MANUAL

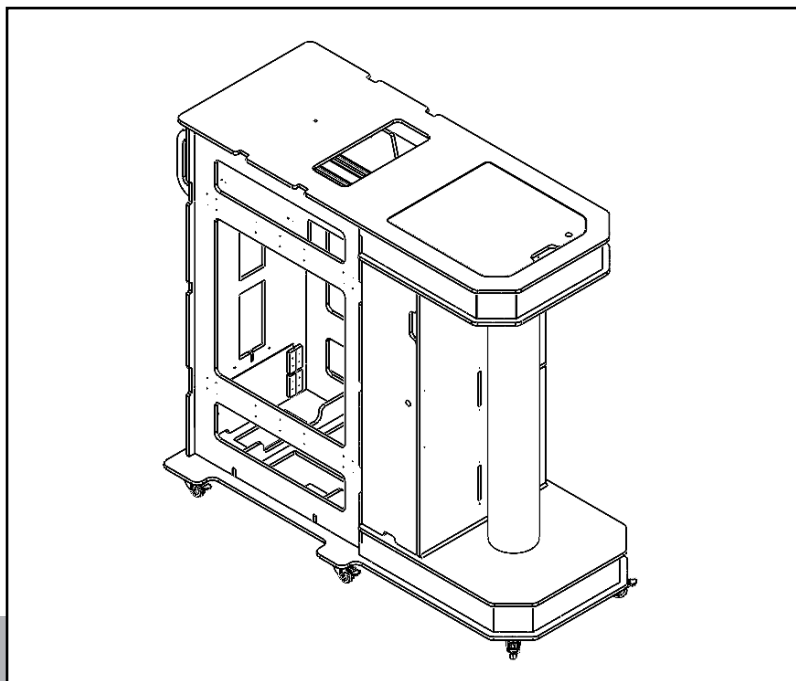

## **SNOEZELEN® SENSTATION**

**22888**

# 01.

**CONTENTS**

# CONTENTS

<b>ABOUT THE PRODUCT .....</b>	<b>4</b>
Product Description.....	5
User Benefits .....	5
Product Compatibility .....	5
<b>TECHNICAL SPECIFICATION .....</b>	<b>6</b>
Included Parts.....	7
Technical Specification .....	7
<b>SAFETY AND CARE.....</b>	<b>8</b>
Product Safety and Care Instructions .....	9
<b>ASSEMBLY &amp; INSTALLATION GUIDE .....</b>	<b>10</b>
Product Assembly Guide/Installation.....	11
Troubleshooting.....	13
<b>USER GUIDE .....</b>	<b>14</b>
User Guide.....	15
<b>FEATS.....</b>	<b>16</b>
<b>NOTES.....</b>	<b>18</b>

02.

**ABOUT THE PRODUCT**

## Production description

Introducing a versatile mobile unit housing Snoezelen® sensory products. Designed with castors and brakes for easy mobility, this unit allows the sensory resources to come to the users, eliminating the need for users to travel to a specific location. It is ideal for settings without a dedicated sensory room.

The mobile unit includes:

- 2 wall panels of your choice (excluding Double Bubble Bonanza and Hip Hop Activity Box)
- Waterless Rainbow Tube (125 cm H)
- Fibre Optics (100 strands, 2 m L) with Light Source
- Milky Way Carpet Mat with Light Source
- Interactive Aroma Panel
- Projector with accessories
- Deluxe 8 Colour Wirefree Controller
- Storage cupboard, trays, and shelves

## User Benefits

This comprehensive solution ensures a flexible and engaging sensory experience for all users.

This portable unit offers a space-saving design, making it perfect for areas with limited room.

It includes many popular Rompa® products, providing a comprehensive sensory experience.

The unit also features ample storage with cupboards, trays, and shelves to keep everything organised. It stimulates multiple senses, engaging sight, touch, hearing, and smell to create an enriching sensory environment.

## Product Compatibility

The unit is compatible with other deluxe wall panels and additional aroma kits for use with the Interactive Aroma Panel. It also comes with a selection of sensory products designed to fit neatly in the drawers and on the top of the SenStation™, enhancing the overall sensory experience.

# TECHNICAL SPECIFICATION


03.

# Included Parts

The unit comes with various shelves designed for organization and storage. It includes one shallow tray, typically green, measuring 38 cm (L) x 28 cm (W) x 7 cm (H). Additionally, there is one medium tray, typically orange, with dimensions of 38 cm (L) x 28 cm (W) x 14 cm (H).

# Technical Specification

The overall size of the unit is 150 cm (L) x 70 cm (W) x 135 cm (H). It features a UK plug with a 13 amp fuse. For mobility, the unit is equipped with four castors, one at each corner, each with a brake that can be pressed down to secure the unit in place.

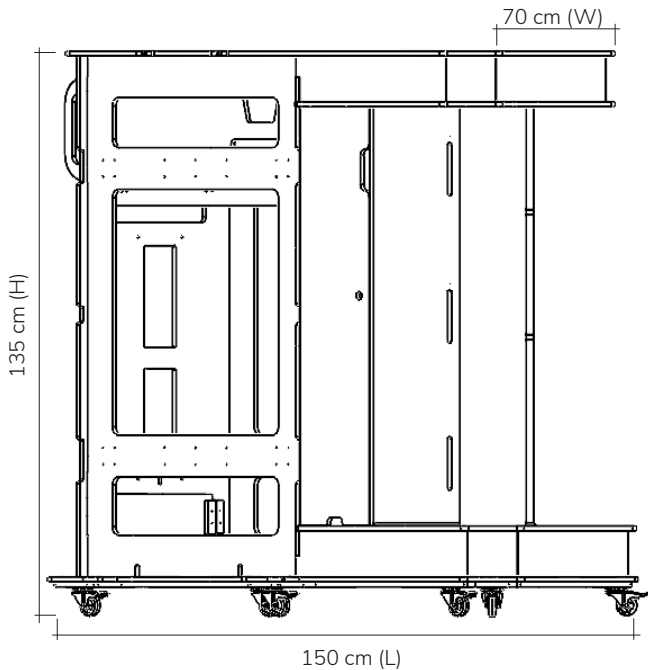

# 04.

**SAFETY AND CARE**




# Product Safety and Care Instructions

Always supervise the use of this product to ensure safety.

Use the brakes on the castors to keep the Snoezelen® SenStation™ securely in place. When moving the unit, make sure all four brakes on the castors are fully open to avoid injury. This product is heavy; if in doubt, seek assistance from a second person to help move and manoeuvre the unit.

Do not sit or stand on the Snoezelen® SenStation™ as it is designed for moving products, not people. Additionally, avoid leaning against the Snoezelen® SenStation™ or placing heavy items against it when not in use.

Front view

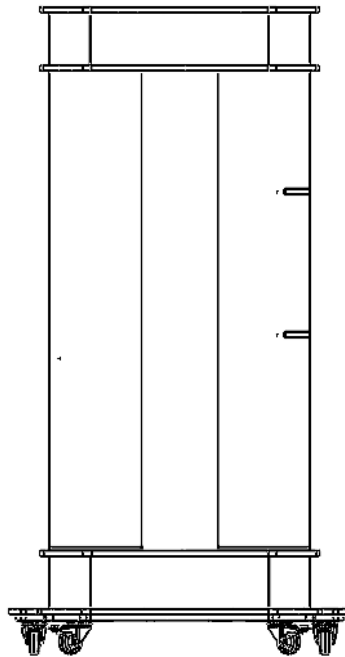

# ASSEMBLY & INSTALLATION GUIDE

05.

# Product Assembly Guide/Installation

This product is supplied ready-assembled, except for the projector. Upon receipt, carefully remove the unit from the crate (two people will be required) and place it on a flat, stable floor.

## 1. Projector Assembly:

- Unpack the projector from its box.
- The base plate for the projector will already be attached to the top of the Snoezelen® SenStation™.
- Attach the projector bracket to this plate.
- Connect the power lead by the plate to the socket at the back of the projector.

## 2. Preparation:

- Remove the protective film from the mirror behind the Waterless Rainbow Tube.
- Plug in any additional products into unused sockets inside the Snoezelen® SenStation™ as necessary.
- Access to the interior of the SenStation™ can be gained by:
  - » Removing one of the wall panels.
  - » Removing the drawers above the Interactive Aroma Panel.
  - » Using the locked doors.

## 3. Socket Locations:

- The individual sockets for each product are located on the vertical wood behind the Waterless Tube.

## 4. Milky Way Mat Setup:

- Take the Milky Way Mat out of the tall cupboard on one of the long sides.
- Place the mat on the floor, close to the SenStation™.

## 5. 8 Colour Controller Setup:

- The 8 Colour Controller will be packed in its box inside the crate.
- Place the controller on its side in the recess at the top of the unit.

## 6. Powering the Unit:

- Plug in the Snoezelen® SenStation™. The plug is located at the bottom of the door on one of the short sides (the side with the Interactive Aroma Panel).
- Ensure the cable comes out of the gap in the door and is not trapped.

## 7. Switching on the Products:

- Use the switches on one of the long sides, towards the top, to switch on individual products.
- Each switch has illustrations to indicate which product it controls (e.g., two switches, one for each of the Deluxe Wall Panels).
- Note: Two switches may be labelled with a projector image. One controls the projector on the top of the unit, while the other is for an additional product (not included).


## 8. Labelling for Ease of Use:

- The individual sockets inside the unit are labelled with the same illustrations as the white switches on the long side, making it easy to match each socket with its corresponding switch (e.g., the socket for the Waterless Tube matches the corresponding white switch for that product).

## 9. Final Touches:

- Fill the trays and shelves with suitable products. Refer to the Technical Specification to determine what can fit inside these.

Side view

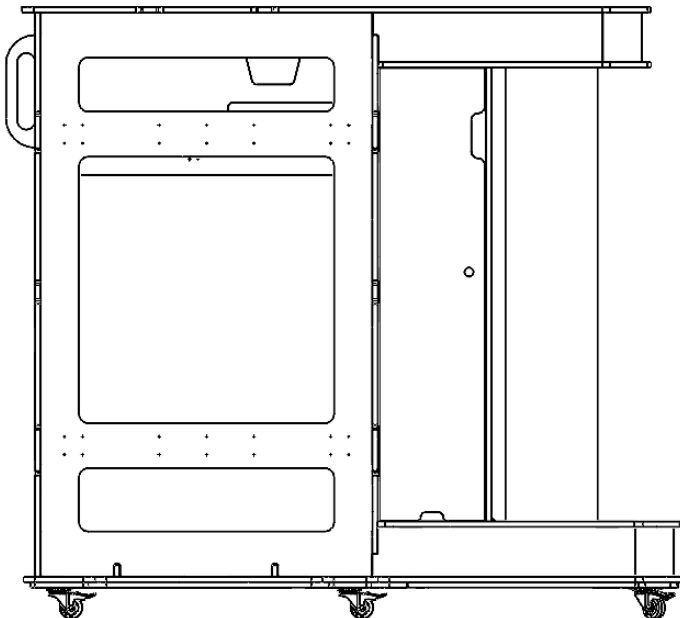

# Troubleshooting

Occasionally, you may encounter issues with the Snoezelen® SenStation™ that require attention. This troubleshooting guide provides solutions to common problems, ensuring optimal performance and functionality of your sensory unit. Follow these steps to address any concerns effectively and maintain a seamless sensory experience for users.

## Common Issues and Solutions:

### 1. Problem: Individual Electrical Products Not Working

Check that the product is properly plugged into one of the sockets inside the unit and that the corresponding ON/OFF switch is set to ON.

Ensure the power lead is plugged into a wall socket and switched ON.

Verify the 13 Amp fuse in the plug.

### 2. Problem: Cloudy Mirror Behind the Waterless Tube

Remove the protective film from the mirror before use.

### 3. Problem: Milky Way Mat Does Not Light Up

Ensure the boss from the mat's fibre optic cable is securely plugged into the light source located between the backs of the Deluxe Wall Panels. Access may require temporarily removing a Deluxe Wall Panel.

### 4. Problem: Fibre Optic Strands Near the Waterless Tube Do Not Light Up

Check that the boss from the fibre optic harness is securely plugged into the light source above the Waterless Tube. Ensure the power lead at the back of the light source is firmly connected.

### 5. Problem: Waterless Tube Does Not Light Up

Check that the power cable is connected to the chassis of the Waterless Tube as indicated by the red sticker on the base. The power supply for the Waterless Tube is located inside the unit, next to the Light Source for the Milky Way Mat.

### 6. Problem: Waterless Tube Does Not React to Its Controller

Check the battery of the controller; if needed, replace.

If the controllers display say "oor" this mean it is out of range, to reconnect you can leave it for 60 seconds and use your other ROMPA® wifi products in the mean time.

06.

**USER GUIDE**

# User Guide

This comprehensive guide is designed to help you maximize the benefits and functionality of your sensory unit. Whether you are setting up, operating, or troubleshooting, this guide provides clear instructions to ensure a smooth and enriching sensory experience. Discover how to utilise each feature effectively and maintain your SenStation™ for long-lasting enjoyment and utility.

- **Moving the Unit:**

When moving the unit, ensure the brakes on the castors are released. Use the integral handles located at the short end to push the unit; do not attempt to lift it. Avoid running the power cable over by the castors.

- **Safety & Care:**

Wipe the trolley clean with a soft duster. For stubborn marks, use a soft cloth slightly dampened with water. Do not use bleach or solvents.

- **Handling Trays:**

Take care not to forcefully pull the trays out. The trays have a locking mechanism on the runners to prevent accidental removal. To remove a tray, gently pull it out as far as it will go. Then, angle the tray to 45° and gently lift it upwards to release it from the runner.


- **Charging the 8 Colour Controller:**

The 8 Colour Controller requires regular charging. Use the additional socket inside the unit with the supplied charger. Pass the charger cable through the void above the Waterless Tube to prevent user access. Alternatively, charge the controller using a separate wall socket when needed.

# FEATURES

08. 



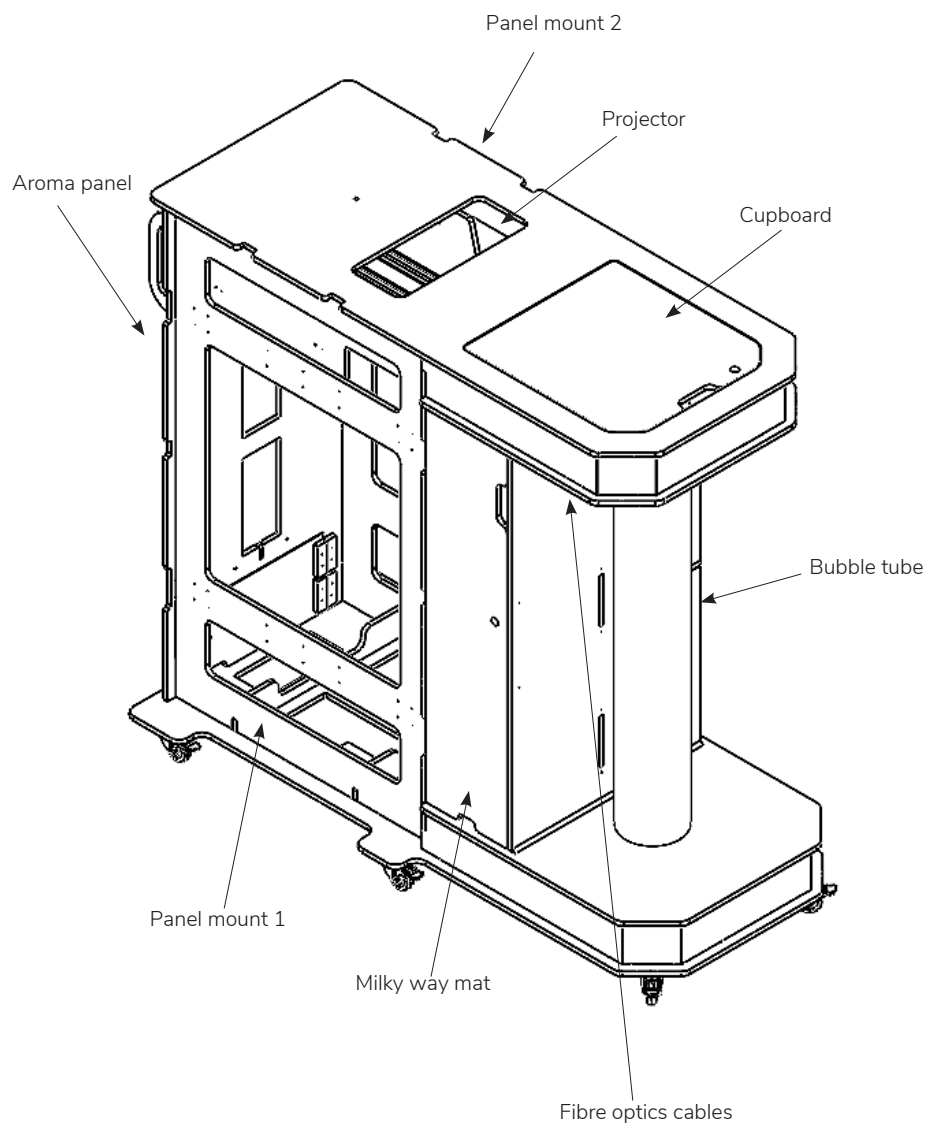

# NOTES

07.



**snoezelen**<sup>®</sup>



Life Changing Sensory Solutions For All

**Rompa<sup>®</sup> Limited, Goyt Side Road,  
Chesterfield, S40 2PH**

**[customer.service@rompa.com](mailto:customer.service@rompa.com)**

**[www.rompa.com](http://www.rompa.com)**

**22888\_07042025**