



Included Parts

- 1 x Snoezelen® Relaxer bubble tube complete with base
- 1 x Bubble tube cap
- 1 x Power supply (black box)

Technical specifications

Size	130cmH x 10.5cm tube diameter, base 36cm diameter x 12cmH
Input	100-240V (47-63Hz)
Output	12V DC at 1.1A
Optimal rating	5W 12V AC

Description

For passive, non-interactive use only. As this tube is not intended to be handled, a bubble tube collar is not required. Ideal for those who wish to enjoy the bubble tube without switch interaction.

Care and Maintenance

- Regularly change the water and use a suitable water-cleansing agent, such as BCB fluid.
- Wash the exterior of the Bubble Tube with a slightly soapy cloth. Wipe excess soap away with a damp cloth.
- Never use solvent based cleaning products, methylated spirits, alcohol wipes or liquid detergents to clean the Bubble Tube.
- This product contains LEDs not lamps – it is highly unlikely that there will be any maintenance required to the LEDs.
- To prevent the growth of microbes in the water change the water monthly, empty the tube altogether if it is not to be used for long periods and do not add bleach, coloured dyes or salt to the water.
- The environment will affect how often the water needs to be changed. Bubble tubes in dark, damp environments with minimal ventilation will need changing at a different frequency to those in bright, well-ventilated rooms.

- Plan for holidays and if the bubble tube will be switched off for long periods, empty the water out.
- If necessary, for example if the tube is used in an area where lime-scale in water is a problem, it may be advisable to use an appropriate water softener.



WARNING: Failure to cleanse your bubble tube water and/or use an appropriate water softener can block bubble tube valves and encourage product failure.

Safety

- Use inside, in dry environments.
- Remove all packaging and dispose of it responsibly. Plastic bags and wrapping can be hazardous to children and vulnerable people.
- Do not attempt to move the bubble tube while filled with water.
- Take care when filling / emptying the bubble tube to ensure the electronics remain dry.
- Always disconnect the power cables when filling / emptying the bubble tube.
- Only use the power supply provided.




IMPORTANT

Always supervise the use of this product

Installation

- Remove the packaging
- Position the product on a flat, stable floor that is capable of holding the weight of the product.
- Plug the power lead into **A**.
- Fill with clean water carefully until the required level is achieved. This should be approximately 5cm (2 inches) from the top of the tube. The water level will rise when switched on. Make sure that the jug or bucket you are using to fill the tube is also clean.
- Put the bubble tube cap (lid) on top of the bubble tube.
- Once the power lead is plugged in at the wall, the green ON indicator light **B** will be illuminated.
- Adjust the intensity of the bubbles using the INTENSITY knob **C**. There are limits on how far it can be turned. Please note that as the intensity of the bubbles increases (the further clockwise it is turned), so will the background noise from the integral pump.

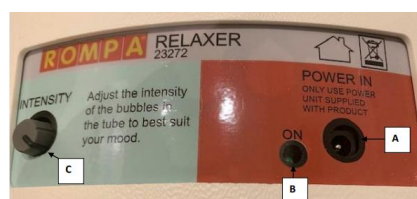

INSTRUCTIONS

23272 SNOEZELEN® RELAXER BUBBLE TUBE

ROMPA®

EN

To empty the water

1. Remove the cap (lid, at the top of the tube).
2. Using an appropriate pump, remove the water from the top of the tube into a bucket. Our rapid change water pump is ideal for this and can be purchased separately. Please note that our bubble tube filling pump is not compatible with this product.

Troubleshooting

In the unlikely event of product failure, please check the following:

No lights, no bubbles

- Check that the power supply ON/OFF switch is switched to ON.
- Check that the mains plug socket is switched on.
- Check that the green ON light is illuminated.