




Life Changing Sensory Solutions For All

snoezelen[®]

INSTRUCTION MANUAL

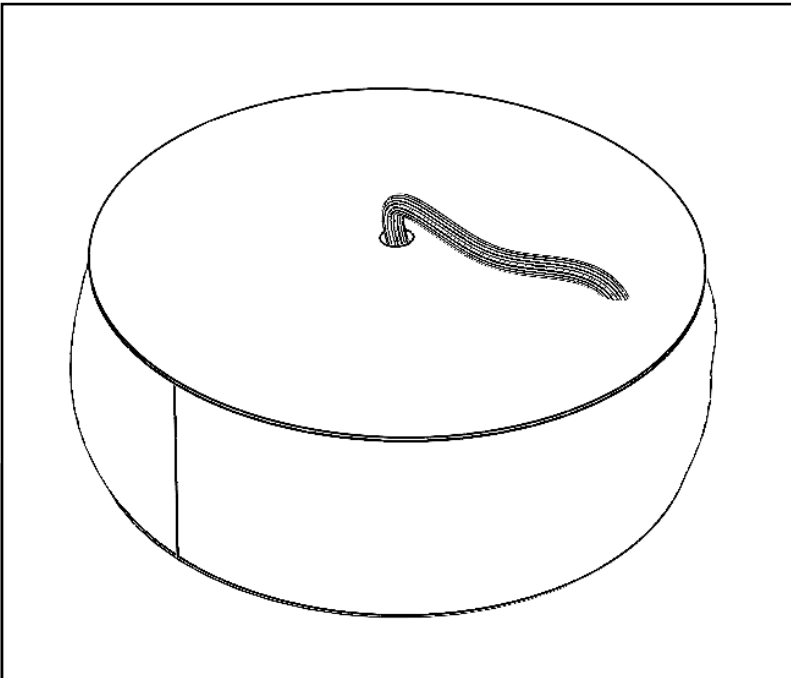

SHIMMERING ISLAND

23642

01.

CONTENTS

CONTENTS

ABOUT THE PRODUCT	4
Product Description.....	5
User Benefits	5
Product Compatibility	5
TECHNICAL SPECIFICATION	6
Included Parts.....	7
Technical Specification	7
SAFETY AND CARE.....	8
Product Safety and Care Instructions	9
ASSEMBLY & INSTALLATION GUIDE	10
Product Assembly Guide/Installation.....	11
User Guide – Functions.....	11
FEATURES	12
NOTES.....	14

02.

ABOUT THE PRODUCT

Product Description

This soft and inviting island is designed to provide a comfortable space for individuals to lie down and relax. The cushioned surface gently contours to the body, allowing users to sink into it, creating a sense of support and calm.

At its centre, vibrant fibre optic lights sprout upward, gracefully cycling through an array of soothing colors. The lighting effects can also be adjusted to match individual preferences using compatible Rompa® products, adding a customisable sensory experience.

As users interact with the delicate fibre optics, they are enveloped in a calming visual and tactile sensation, making the island a perfect addition to any sensory environment.

User Benefits

- This product stimulates a variety of senses, including sight, touch, and the vestibular system, making it a comprehensive sensory experience.
- It creates a calming environment that encourages users to relax and unwind, promoting a sense of well-being.
- It empowers the individual by providing control over the lighting effects, allowing them to tailor the visual experience to their preference.
- The fibre optics offer gentle, weighted stimulation, further enhancing the tactile experience and contributing to the overall therapeutic benefits.

Product Compatibility

This product is compatible with several sensory accessories, including internal light sources and fibre optics (FO) integrated within the design. For precise colour control, it works seamlessly with a range of Rompa® products, such as the Colour Command Centre™ Panel, the Deluxe 8-button Controller, and interactive tools like the Talking Cube and Mini Talking Cube. Additionally, it can be paired with the Wireless Colour Controller Cushion Wall Mat to further enhance the customisation options and sensory engagement.


TECHNICAL SPECIFICATION

03.

Included Parts

The product package includes a PVC structure filled with soft foam cubes for added comfort and support, an integrated light source to create captivating visual effects, a sturdy wooden box for housing the components, and delicate fibre optic strands that provide both tactile and visual sensory stimulation.

Technical Specification

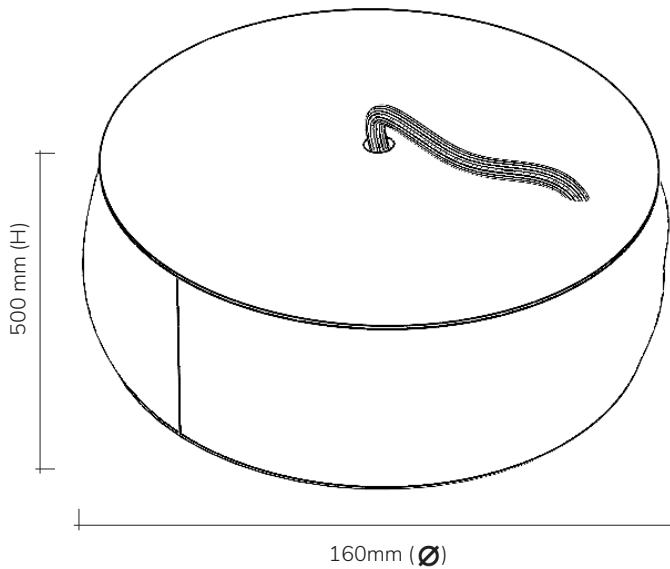

04.

SAFETY AND CARE

Product Safety and Care Instructions

This product should always be used under supervision to ensure safety. Ensure that the light source is switched off when not in use, and avoid leaving the unit powered on unattended to prevent any potential hazards or unnecessary wear.

PVC:

- Use a soft brush remove all loose dust/dirt. Clean with a mild detergent (such as Lux soap flakes in luke-warm solution) applied to a sponge or damp cloth.
- This may be sanitised using a weak solution of Milton fluid.
- Do not use excessive amounts of water and always ensure the PVC is dry after cleaning.
- Do not use an abrasive cleaning pad.
- Do not use boiling/very hot water, washing up liquid / aggressive detergent or solvent based cleaner as this will cause the coating to discolour and go hard.
- Steam cleaning is not advised.



ASSEMBLY & INSTALLATION GUIDE

05.

Product Assembly Guide/Installation


- Feed the fibre optics into the top of the soft island, with the connection point going in first.
- Once the connection point appears at the bottom, attach the Rompa® Light Source, ensuring it fits securely into the wooden box provided.
- Insert the wooden box containing the light source into the cut-out section of the Shimmering Island.
- Connect the power supply to the light source and plug it into the mains.
- Make sure the light source is turned on.
- Carefully turn the island the right way up.

USER GUIDE - FUNCTIONS

Compatible with any product that can control Rompa® Light sources

FEATURES

06.

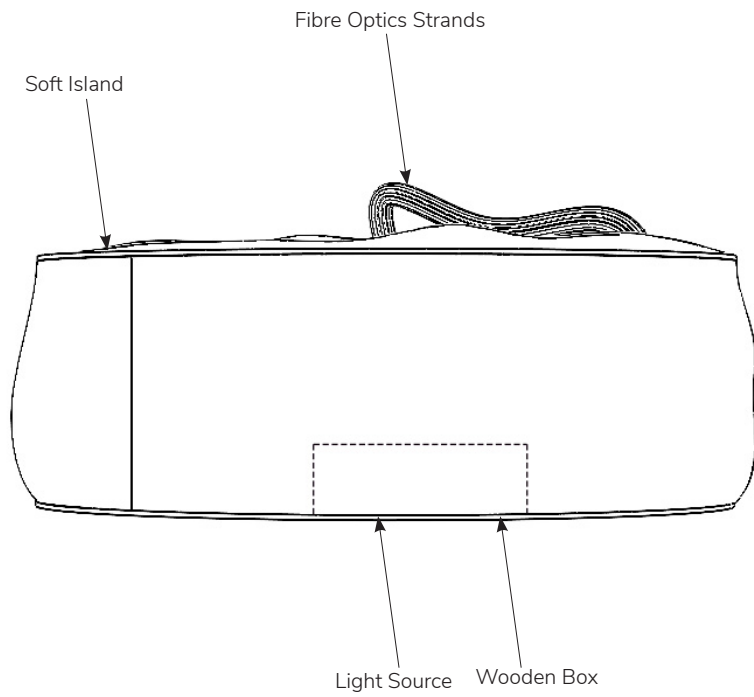

NOTES

07.

snoezelen[®]



Life Changing Sensory Solutions For All

**Rompa[®] Limited, Goyt Side Road,
Chesterfield, S40 2PH**

customer.service@rompa.com

www.rompa.com

23642_280125