

Warranty and Terms of Use
Suspended platform
Product Code: 110M2



TABLE OF CONTENTS

INTENDED USE	2
WARNINGS	3
PRECAUTIONS.....	3
TERMINOLOGY AND EXPLANATION OFSYMBOLS	4
MAINTENANCE	4

INTENDED USE

Therapy aids (platforms) are equipment used for exercise during Sensory Integration therapy and are designed for therapeutic exercises for children with autism, ADHD, dyslexia, cerebral palsy, mental retardation, X chromosome fragility, Down syndrome, Children with Sensory Integration Disorders.

This equipment, thanks to the possibility of any configuration, is especially used for the development of motor planning mechanisms in dyspraxia and the development of equivalent reactions.

This product may only be used in the presence of a person who has completed a specialized course in Sensory Integration therapy.

Maximum load – 90 kg.



WARNINGS

Before each class, the condition of the product (visible tears, abrasions, damage) and hooking elements (swivels, swivels, eyelets) should be checked

- In case of any damage or abrasion observed (applies to moving parts, metal-to-metal friction), please contact the selling company.
- In the case of using a single-hook product, it should be absolutely attached to the swivel.
- The exerciser may need belay when starting the exercise
- During therapy in which a slide will be used, the therapist should belay the person exercising at all times.

SAFETY PRECAUTIONS

The safety of your customers depends on:

- appropriately selected and appropriate activities tailored to their physical capabilities
- work with proper and safe techniques
- a properly planned work area
- proper use of equipment
- avoid exposing the equipment to external factors (i.e. ignition, tearing, tearing, washing with inappropriate cleaning agents) that may weaken the entire structure of the device
- proper maintenance of the device

Platforms are not intended for:

- use outside the room
- use without the consent, knowledge and presence of a specially trained therapist in the field of Sensory Integration Therapy or another person who has undergone a special course in Sensory Integration
- recreational use
- use it in a different way than it is used in Sensory Integration therapy

It is forbidden!

- Use in a manner other than intended
- violate the materials in any way
- self-repairs

TERMINOLOGY AND EXPLANATION OF SYMBOLS



Attention! Please refer to the manual

MAINTENANCE

- Keep the equipment in a place where it is not exposed to external factors that may cause damage (avoid moisture, avoid places with increased temperatures, e.g. around a radiator).
- It should be kept in a specially designed wardrobe, for example, to avoid dirt, dust and unnecessary external, natural and mechanical factors.
- The equipment can only be cleaned manually with cleaning agents that do not contain corrosive agents and are harmful to human health
- Equipment can be cleaned with a vacuum cleaner